

EP 600 Combi

Programat[®] EP 3000

Programat[®] EP 5000

Important information regarding the software update!

Congratulation!

You have decided to carry out a software update on your Ivoclar Vivadent combination furnace. Please note the following advice:

Press programs: Please note that all press programs (including individual programs) are reset to the default factory settings during the software update. Therefore, we recommend noting down the settings of individual press programs before the update, since they cannot be saved on a USB stick or PC. Even though **firing programs** are not affected by the update, we advise you to save them on a USB stick or PC (by means of PrograBase2) before conducting the update.

For further information, please refer to the Operating Instructions.