

Programat® CS

Tabla de programas en °C / °F

Programat CS Tabla de programas													Valores en °C				
Validez: a partir del Software V3.20																	
Rango de valores para todos los programas	1 - 20	min.	B	S	t↑	T	H	V1	V2	L	tL	[°C]	[mm:ss]	Hv			
															Temperatura servicio [°C]	Tempo cierre [mm:ss]	Aumento temperatura [°C]
			100	00:18	30	100	00:00	0	0	0	0	0	00:00	00:00			
		max.	700	30:00	140	1200	60:00	1200	1201	1200	50	700	60:00	60:00			
Grupo de materiales	P.Nr.																
IPS e.max CAD	1	Cristal./Glasea.	m.V.	403	06:00	90 / 30	820 / 840	00:10 / 07:00	550 / 820	820 / 840	700	0	0	00:00	00:00		
IPS e.max CAD	2	Corrección	m.V.	403	06:00	90 / 30	820 / 840	00:10 / 03:00	550 / 820	820 / 840	700	0	0	00:00	00:00		
IPS e.max CAD	3	Speed-Cr./Spray	m.V.	-	-	-	-	-	-	-	-	-	-	-			
IPS Empress CAD	4	Color/Glaseado	o.V.	403	06:00	100	790	01:00	0	0	0	0	0	00:00	00:00		
IPS e.max CAD-on	5	Fusión/Cristal.	m.V.	403	02:00	30 / 30	820 / 840	2:00 / 7:00	550 / 820	820 / 840	600	0	403	06:00	00:00		
IPS e.max CAD-on	6	Caract./Glasead.	m.V.	403	06:00	60 / 30	820 / 840	0:10 / 3:00	550 / 820	820 / 840	600	0	0	00:00	00:00		
IPS e.max CAD	7	Imp./Mo./Cristal.	m.V.	403	06:00	60 / 30	770 / 850	0:10 / 10:00	550 / 770	770 / 850	700	0	0	00:00	00:00		
My Programs	8	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	9	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	10	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	11	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	12	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	13	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	14	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	15	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	16	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	17	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	18	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	19	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		
My Programs	20	Individual	o.V.	403	00:18	30	410	00:01	0	0	0	0	0	00:00	00:00		

P.Nr. 1, 2, 3, 5, 6, 7: Programa en 2 etapas m.V. = con vacío
o.V. = sin vacío

Programat CS Tabla de programas													Valores en °F				
Validez: a partir del Software V3.20																	
value ranges for all programs	1 - 20	MIN	B	S	t↑	T	H	V1	V2	L	tL	[°F]	[mm:ss]	Hv			
															Temperatura servicio [°F]	Tempo cierre [mm:ss]	Aumento temperatura [°F]
			212	00:18	54	212	00:00	0	0	0	0	0	00:00	00:00			
		MAX	1292	30:00	252	2192	60:00	2192	2194	2192	90	1292	60:00	60:00			
Material group	P.Nr.																
IPS e.max CAD	1	Cristal./Glasea.	m.V.	757	06:00	162 / 54	1508 / 1544	00:10 / 07:00	1022 / 1508	1508 / 1544	1292	0	0	00:00	00:00		
IPS e.max CAD	2	Corrección	m.V.	757	06:00	162 / 54	1508 / 1544	00:10 / 03:00	1022 / 1508	1508 / 1544	1292	0	0	00:00	00:00		
IPS e.max CAD	3	Speed-Cr./Spray	m.V.	-	-	-	-	-	-	-	-	-	-	-			
IPS Empress CAD	4	Color/Glaseado	o.V.	757	06:00	180	1454	01:00	0	0	0	0	0	00:00	00:00		
IPS e.max CAD-on	5	Fusión/Cristal.	m.V.	757	02:00	54 / 54	1508 / 1544	2:00 / 7:00	1022 / 1508	1508 / 1544	1112	0	757	06:00	00:00		
IPS e.max CAD-on	6	Caract./Glasead.	m.V.	757	06:00	108 / 54	1508 / 1544	0:10 / 3:00	1022 / 1508	1508 / 1544	1112	0	0	00:00	00:00		
IPS e.max CAD	7	Imp./Mo./Cristal.	m.V.	757	06:00	108 / 54	1418 / 1562	0:10 / 10:00	1022 / 1418	1418 / 1562	1292	0	0	00:00	00:00		
My Programs	8	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	9	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	10	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	11	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	12	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	13	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	14	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	15	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	16	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	17	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	18	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	19	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		
My Programs	20	Individual	o.V.	757	00:18	54	770	00:01	0	0	0	0	0	00:00	00:00		

P.Nr. 1, 2, 3, 5, 6, 7: Programa en 2 etapas m.V. = con vacío
o.V. = sin vacío

Versión: 4
Fecha de edición: 2014-09
Validez: a partir del Software V3.2

Impresa en Austria
© Ivoclar Vivadent AG, Schaan/Liechtenstein
604557/2014-09/s

ivoclar
vivadent®