# Virtual<sup>®</sup>

## Tray Adhesive

### EN Instructions for Use

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# English

## Description

Self-curing, single-component adhesive for silicone-based impression materials.

## Composition

Virtual Tray adhesive contains polydimethylsiloxane, polysiloxane resin and solvent

## Indication

Virtual Tray Adhesive is specifically intended for the adhesion of cured addition-reaction silicone (vinylpolysiloxane) impression materials to standard metal and plastic trays.

## Contraindication

- If the patient is known to be allergic to any of the ingredients in Virtual Tray Adhesive.
- If the tray cannot be properly cleaned and dried.

## Side Effects

Virtual Tray Adhesive may cause a sensitization reaction in patients with a hypersensitivity to any of its ingredients. In these cases, the material should no longer be applied.

## Interactions

None

## Application

1. Be sure that all surfaces of the impression tray are oil-free, clean and dry.  
**Note:** If the disinfection of the impression tray cannot be ensured, the tray adhesive must be filled into a dappen glass, for example, and applied using a disposable brush.
  2. Apply a thin layer of Virtual Tray Adhesive using the brush provided to all surfaces of the impression tray (metal or plastic) that will come into contact with the impression material.
  3. Allow the tray adhesive to dry for approximately 3 minutes (or blow dry with compressed oil-free air).
  4. Replace bottle cap immediately.
- **Removal of Tray Adhesive / Cleaning of Impression Trays**  
Tray Adhesive residue can be removed with alcohol.

## **Special Note**

None

## **Warnings**

- Contains Xylene, hazardous to health
- Avoid contact between Virtual Tray Adhesive and skin, mucous membrane, and eyes; if contact should occur, rinse thoroughly with running water (for 10 minutes, with open palpebral fissure). Consult ophthalmologist. If the material comes into contact with the skin, immediately wash affected areas with water.
- Do not inhale vapor.
- Close container after use.
- Keep the product away from flame and sparks.
- Attention: May stain clothing

## **Storage**

- Do not use Virtual Tray Adhesive after the indicated date of expiration.
- Close bottle immediately after use.
- Do not use material if the individual components show clear signs of change of consistency.
- Storage temperature: 2 – 28° C (36 – 82° F)
- Shelf life: See expiration date on label and packaging.

## **Keep out of the reach of children!**

### **For use in dentistry only!**

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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### **Ivoclar Vivadent Limited**
### **Ivoclar Vivadent, Inc.**