Virtual[®]

CADbite Registration

EN Instructions for Use

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For dental use only.

Description

Virtual CADbite Registration is an addition-reaction silicone developed specifically to record the occlusal bite for the fabrication of indirect restorations. The material offers a thixotropic viscosity, high precision, a fast setting time with a final hardness of 32 Shore D and is dimensionally stable. Virtual CADbite Registration is reflective and may be used for the optical recording of data when fabricating restorations using CAD/CAM technology (antagonist bite registration).

- Fabrication of bite registrations that help dental professionals ensure proper articulation of the model when creating indirect restorations.
- Optical recording of the antagonist situation for CAD/CAM systems

Colours and working times

Technical data

Colour	Blue
Mixing ratio [base:catalyst]	1:1
Working time after mixing measured at 23 °C (73 °F)	max. 30 sec.
Set time in mouth	min. 45 sec.
Detail reproduction	2 µm
Linear Dimensional Change [24h]	< 0.05%
Shore hardness after polymerization	32 Shore D

Indications

Impressions of dentulous and edentulous jaw sections within the framework of restorative, prosthetic, orthodontic or functional treatment measures.

Contraindications

- The use of Virtual CADbite Registration is contraindicated if the patient is known to be allergic to any of its ingredients.

Limitations of use

The product is not intended to be reprocessed or reused.

Side effects

There are no known side effects to date.

Interactions

The setting of vinylpolysiloxanes is inhibited by latex gloves.

Do not touch teeth, preparations or retraction cords with latex gloves. It is recommended that operators wash their hands thoroughly or use vinyl gloves in order to eliminate all traces of impurities. Other products which may inhibit the set of impression materials include rubber dams, retraction cords and certain substances.

If the operator suspects that the preparation has been contaminated, the preparation must be rinsed and dried to eliminate all traces of impurities.

Composition

– Virtual CADbite Registration

Silicon dioxide, vinylpolysiloxane, inorganic aluminium filler (CAS 1318-02-1), methyl hydrogen siloxane, titanium dioxide

Application

– Loading the cartridge dispenser

Fig. 1

Fig. 2

Fig. 3

Fig. 4

1. Press the black release lever located below the plunger on the back of the dispenser and pull plunger as far back as possible (Fig. 1).
2. Lift the cartridge lock and insert the cartridge with the "V" shape on the cartridge base turned downwards. Lower the cartridge lock (Fig. 2).
3. Remove cartridge cap by turning it 1/4 turn counter clockwise (Fig. 3). Discard the cap.
4. Important: It is very important to clear or bleed the cartridge prior to applying the mixing tip! This will assure a proper mix and set time. Gently press on the dispenser lever until both components (base and catalyst) begin to flow out of the cartridge evenly onto a mixing pad.
5. Attach a mixing tip. Push the tip down until the notch on the mixing tip touches the notch on the cartridge. While gripping the coloured base, and not the mixing tip, turn the mixing tip base 1/4 turn clockwise (Fig. 4).
6. Begin mixing by pressing gently on the lever of the dispenser.

– Conventional bite registration

1. Dispense bite registration material directly onto the occlusal surface of the lower teeth.
2. The patients should be asked to close into their normal occlusal relationship and hold the position until the material is completely set (approx. 45 seconds).

– **Optical data recording**

1. Dispense Virtual CADbite Registration directly onto the occlusal surface of the prepared teeth.
2. The patient should be asked to close their teeth together without any functional movement and hold the position until the material is completely set (approx. 45 seconds).
3. If required, the bite registration can be removed and trimmed to enable data recording.
4. Then reseat the bite registration on the preparation and take an image of the antagonist situation.

– **Removing the cartridge**

1. Lift the release lever to pull the slide back. Lift the cartridge lock and remove the cartridge.
2. Leave the mixing tip on the cartridge. The material will polymerize within the mixing tip, functioning as a natural cap.

– **Disinfection**

Impressions made with Virtual impression materials can be immersed in a disinfectant solution (e.g. glutaraldehyde 0.5%, benzalkonium chloride 0.5%). Please observe the instructions of the manufacturer. The disinfection does not affect the surface or the dimension.

Special notes

Virtual CADbite Registration should be processed at room temperature (23 °C/73 °F). Lower temperatures, e.g. when stored in the refrigerator, will prolong the working time and the time the impression material should remain in the mouth, while higher temperatures will reduce it.

Polysiloxanes are chemically resistant.

Unpolymerized material may stain clothing.

Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: www.ivoclarvivadent.com, and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website (www.ivoclarvivadent.com).

Disposal information

- Remaining stocks must be disposed of according to the corresponding national legal requirements.

Shelf life and storage

- Storage temperature 2-28 °C (36-82 °F)
- Do not use the product after the indicated date of expiration.
- Expiration date: See note on cartridge

Additional information

Keep material out of the reach of children!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

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