# IvoBase<sup>®</sup> Injector

Short Instructions | Kurzanleitung | Mode d'emploi : l'essentiel  
Istruzioni d'uso – versione ridotta | Guía rápida | Instruções Resumidas

CE

## Dear Customer

Thank you for having purchased the IvoBase Injector. In order to render the setting-up and the initial use of the device as easy as possible, we have prepared these short instructions.

**These short instructions by no means replace the detailed Operating Instructions. We therefore kindly ask you to thoroughly read the Operating Instructions and follow the notes contained therein.**

Enjoy working with the IvoBase Injector.

# 1. Initial start-up

## Step 1: Unpacking and choosing the appropriate location

Unpack the Injector (please follow the instructions on the outer packaging) and set it on a suitable flat surface.

## Step 2: Removing the transport protection

Remove the transport protection on the safety door and the waste water container.


## Step 3: Power connection

Please make sure that the voltage indicated on the rating plate complies with the local power supply. Please also refer to the technical information contained in the Operating Instructions (Chapter 9). Now connect the power cord with the power socket and the wall socket.


## Step 4: Switching on the Injector

Put the power switch to position I. The Injector then conducts an automatic self-test. Make sure that the safety door is closed during the test. The standby display appears after the self-test has been successfully completed. Now the Injector is ready for use.


If the self-test has not been successful, an error message appears.  
(For more details, see Chapter 8 in the Operating Instructions.)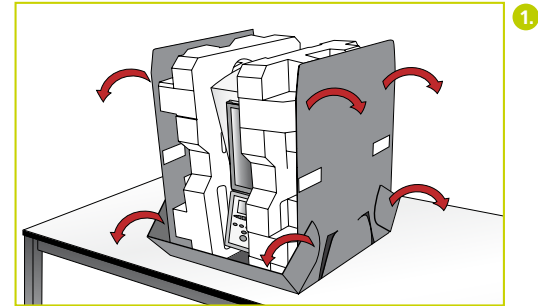


1.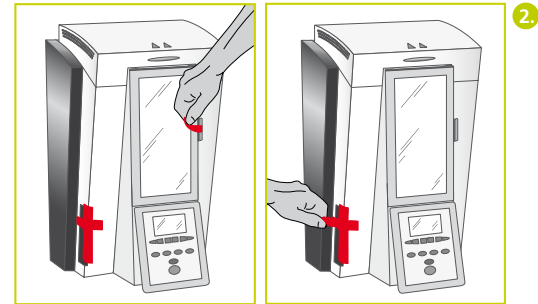

2.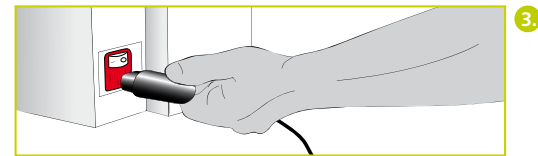


3.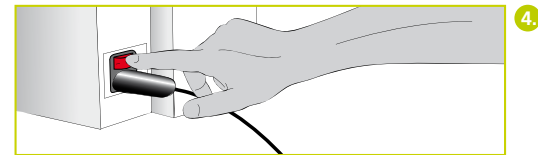

4.

## 2. Operation

The IvoBase Injector is operated/controlled by means of the membrane-sealed keypad. The most important keys at a glance:

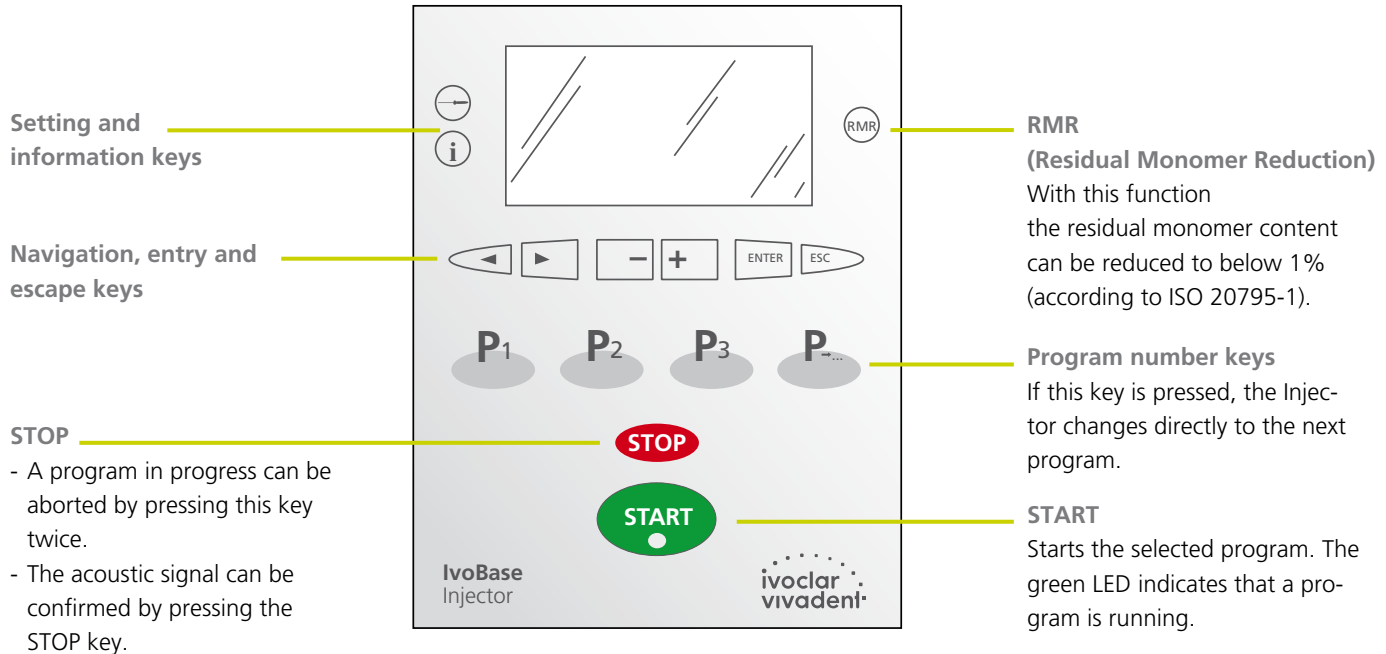### 3. Carrying out the injection process

#### Step 1:

Select the desired program with the program number keys.

#### Step 2:

Open the safety door.

#### Step 3:

Slide the flask into the holder intended for this purpose as shown in the picture.


#### Step 4:

Close the safety door. A program cannot be started if the safety door is open.

Press the Start key to start the program.

#### Step 5:

At the end of the program, an acoustic signal sounds. Remove the flask and start the divestment, as described in the Instructions for Use of the material.

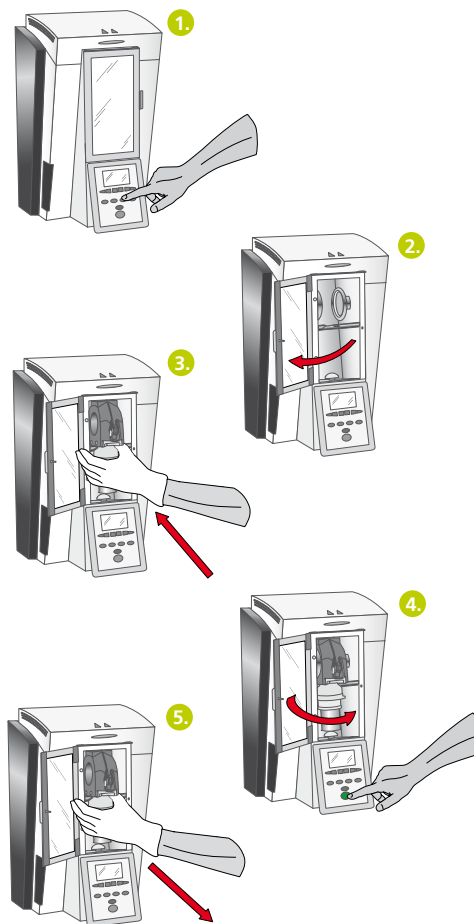Always use the thermal glove supplied when removing the flask from the Injector!

## 4. Program overview

Program No.	Name	Description	Duration
1	IvoBase Hybrid	Program for the polymerization of the IvoBase Hybrid material, residual monomer $\leq$ 1.5%	35 min.
		Program for the polymerization of the IvoBase Hybrid material with additional RMR, residual monomer $\leq$ 1.0%	45 min.
2	IvoBase High Impact	Program for the polymerization of the IvoBase High Impact material, residual monomer $\leq$ 1.5%	50 min.
		Program for the polymerization of the IvoBase High Impact material with additional RMR, residual monomer $\leq$ 1.0%	60 min.
3	SR Ivocap High Impact	Program for the polymerization of the SR Ivocap High Impact material, residual monomer $\leq$ 1.5%	55 min.
		Program for the polymerization of the SR Ivocap High Impact material with additional RMR, residual monomer $\leq$ 1.0%	65 min.
4	SR Ivocap Clear	Program for the polymerization of the SR Ivocap Clear material, residual monomer $\leq$ 1.5%	55 min.
		Program for the polymerization of the SR Ivocap Clear material with additional RMR, residual monomer $\leq$ 1.0%	65 min.
5	SR Ivocap Elastomer	Program for the polymerization of the SR Ivocap Elastomer material	65 min.

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## **Ivoclar Vivadent Limited**
## **Ivoclar Vivadent, Inc.**