# bluephase® 20i

Licence to cure

new

LED for every use

What if...	2
Was ist, wenn...?	3
Que faire en cas de...?	4
Cosa fare se....?	5
¿Qué sucede si....?	6
E se ...?	7
Vad är fel när ...?	8
Hvad nu hvis ...?	9
Mitä jos...?	10
Hva er feil når ...	11
Wat te doen als ...?	12
Τι κάνω εάν ...;	13
Что делать, если	14

CE 0123

For dental use only!

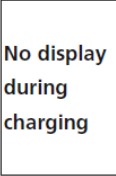
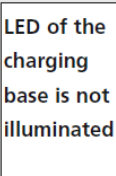
Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist

Made in Austria

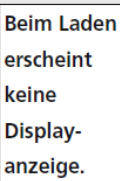
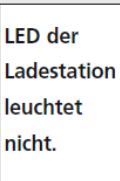
Ivoclar Vivadent AG, FL-9494 Schaan / Liechtenstein

ivoclar  
vivadent  
clinical

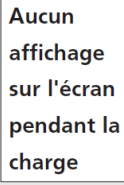
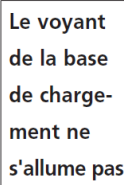
## What if...

Symbol	Causes	Error rectification
	Electronic defect in the handpiece or battery	Remove and reinsert the battery. If the error persists, replace the battery with the power pack (Click & Cure). If the error persists, please contact your dealer or your local Service Center.
	Apparatus is overheated or under-cooled	Allow the apparatus to cool down (or assume room temperature if it is undercooled) and try again after a certain time. If the error persists, please contact your dealer or your local Service Center.
	Electronic defect in the battery	Remove and reinsert the battery. If the error continues to be displayed, place the apparatus into the charging base. If the error persists nonetheless, replace the battery with the power pack (Click & Cure). Please contact your dealer or your local Service Center.
	Battery empty	Position the handpiece in the charging base. "Charging Battery" briefly appears in the display before the display returns to the stand-by mode. If the battery is not charged, the contacts have to be cleaned. If the error persists, please contact your dealer or your local Service Center.
	Charging contacts contaminated Battery in safety mode (integrated protective circuit)	Check if the handpiece has been correctly placed into the charging base. Clean battery contacts. Information: "Charging Battery" briefly appears in the display before the display returns to the stand-by mode. If cleaning the battery contacts does not resolve the problem, remove the battery from the apparatus. Charge the battery separately from the handpiece in the charging base for approximately 10 minutes. If the error persists, please contact your dealer or your local Service Center.
	Power pack not connected or defective	Check the battery contacts and if the power pack is correctly positioned in the charging base or if the power pack is connected to the power supply by means of the power cord (display on the power pack lights up in green if it works correctly). If the error persists, please contact your dealer or your local Service Center.

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