**Description**

OptraSculpt and OptraSculpt Pad are modelling instruments designed for the stick-free adaptation, shaping and contouring of uncured, sculptable direct composite restoratives (e.g. IPS Empress Direct, Tetric EvoCeram Bulk Fill) as well as for the modelling and smoothing of transitions created with laboratory composite in paste form (e.g. SR Nexco Paste).

The perfectly fitting disposable attachments are inserted into the holding ring on the working end of the autoclavable, hand-held instrument. The non-stick effect, which becomes obvious when composite resin is adapted and contoured with the instrument, renders processing of the restorative material quick and efficient.

The OptraSculpt modelling attachments in different shapes have been especially adjusted to meet the requirements of occlusal surface and marginal ridge design based on the natural anatomy. Therefore, they offer a wide variety of uses. The pad attachments for the OptraSculpt Pad instrument are available in two sizes. They facilitate the sculpting and contouring of extensive defects in the anterior region.

Reference scales on the OptraSculpt Pad handle assist in evaluating symmetries, tooth width proportions and alignments when shaping large, demanding Class IV and direct veneer restorations.

**Composition***OptraSculpt:*

- OptraSculpt attachments: Polypropylene and SEBS
- OptraSculpt hand instrument: Stainless steel and PEEK

*OptraSculpt Pad:*

- OptraSculpt Pad attachments: Polyethylene and EVA
- OptraSculpt Pad hand instrument: Stainless steel and PEEK

**Indication**

Adaptation, shaping and contouring of uncured sculptable composite restoratives. OptraSculpt Pad is especially indicated for Class III, IV and V restorations. OptraSculpt is particularly suitable for the sculpting and contouring of Class I, II and V restorations.

**Contraindication**

The products should not be used

- if the stipulated working technique cannot be applied;
- if a patient is known to be allergic to any of the ingredients of OptraSculpt or OptraSculpt Pad.

**Side effects**

None known to date.

**Warnings!**

**The use of a rubber dam is recommended in order to avoid accidental ingestion or aspiration of small parts.**

Do not use OptraSculpt and OptraSculpt Pad for purposes other than intended!

**Application in the dental office**

Select the appropriate OptraSculpt or OptraSculpt Pad attachment, depending on the size of the restoration and the area in which it will be employed. Remove the attachment from the packaging either directly or indirectly (e.g. using sterile tweezers), ensuring freedom from contamination. Then insert the attachment into the ring-shaped end of the hand-held instrument.

The colour-coded OptraSculpt attachments are supplied correctly aligned in refill boxes, so that they can be taken up directly with the instrument.

To attach the disc-shaped OptraSculpt Pad attachments, place them flat on a firm, contamination-free working surface.

# OptraSculpt® & OptraSculpt® Pad

<b>EN</b>	<b>Instructions for Use</b> Composite modelling Instruments	<b>FI</b>	<b>Käyttöohjeet</b> Yhdistelmämuovoin muotoiluinstrumentit
<b>DE</b>	<b>Gebrauchsinformation</b> Modellierinstrumente für Composites	<b>NO</b>	<b>Bruksanvisning</b> Instrumentertil modellering av kompositmateriale
<b>FR</b>	<b>Mode d'emploi</b> Instrument à modeler les composites	<b>NL</b>	<b>Gebruiksaanwijzing</b> Composietmodelleerinstrumenten
<b>IT</b>	<b>Istruzioni d'uso</b> Strumenti di modellazione per composito	<b>EL</b>	<b>Οδηγίες Χρήσεως</b> Εργαλεία διαμόρφωσης σύνθετων ρητινών
<b>ES</b>	<b>Instrucciones de uso</b> Instrumentos para modelar composite	<b>TR</b>	<b>Kullanma Talimatı</b> Kompozit modelaj aletleri
<b>PT</b>	<b>Instruções de Uso</b> Instrumentos para modelagem de compósitos		

Place the disc-shaped OptraSculpt Pad attachment flat and upright on a working surface.

Open the dispenser box by exerting slight pressure on the marked surface.

Slide the ring over the shaft until it meets the base of the pad.

Select the appropriate attachment based on the colour coding described below:  
Anthracite: ball; Turquoise: chisel;  
Purple: point

The OptraSculpt Pad attachment is correctly inserted in the holding ring.

Slide the ring over the shaft of the attachment until the ring sits flat on the attachment. Then remove the attachment which is now firmly attached to the instrument.

### OptraSculpt and OptraSculpt Pad modelling attachments are designed for single use only.


After having applied the uncured, sculptable composite restorative, adapt and shape it with a dabbing motion, applying gentle pressure. After their intraoral use, OptraSculpt and OptraSculpt Pad attachments must be removed and discarded. Subsequently, the hand instrument is cleaned, disinfected and sterilized.

#### Using the reference scale on the instrument handle:

The OptraSculpt Pad instrument features two reference scales.

**Scale 1** provides plain reference markings to compare the tooth-to-tooth width proportions. The horizontal line serves as a reference in the evaluation of the incisal edge line.


**Scale 1** can be aligned with the incisal edge to e.g. compare the contralateral tooth width proportions and/or evaluate the incisal edge line against the horizontal line on the scale.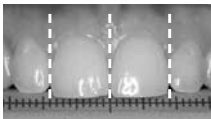

**Scale 2** contains the markings for the ideal average anterior tooth widths (dashed vertical lines) and the typical average position of the tooth axes in the upper jaw (oblique dotted lines). These markings provide orientation in the esthetic reconstruction of extensive anterior defects. The values are relative values that are intended to serve as a guide in the harmonious, anatomical design of anterior restorations. The scale is positioned correctly if the triangle in the middle of the scale points towards the contact point between the central incisors in the upper jaw and the numbers at both sides of the scale are consistent with the quadrants under examination.

#### Scale 2 is unsuitable for use in the mandibular jaw!

**Scale 2** is utilized to compare the clinical situation against the ideal average tooth width proportion and angular alignments in the maxillary anterior dentition. Consequently, esthetic enhancements can be easily planned and areas of improvement identified.

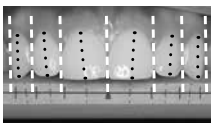**Important:** This scale indicates relative values only. The marked widths and angular alignments should not simply be taken at face value.

Example: The example depicted above shows comparatively ideal tooth width proportions, even if the overall tooth width may appear slightly larger than average. The angles of the lateral incisors and canines, however, are too straight.

#### Application in the dental laboratory

OptraSculpt and OptraSculpt Pad are also suitable for the shaping and contouring of laboratory composites in paste form, e.g. SR Nexco Paste. Due to the pads' non-stick properties, large veneers can be conveniently contoured and given an ideal shape, particularly with the small pad. OptraSculpt and OptraSculpt Pad are a useful addition to the instrument range of the dental laboratory. They are especially useful for the processing of laboratory composites in paste form.

#### Disinfection, cleaning and sterilization

##### General notes

- Clean and disinfect every instrument before each use. It is strongly recommended that the instruments are additionally sterilized in a steam sterilizer.
- Do not use damaged instruments. Only suitably trained staff should carry out repair and maintenance work.

- Do not place the instruments in NaCl solution (to avoid the risk of pitting and/or stress corrosion cracking).
- Protective gloves and goggles complying with the requirements of directive 89/686/EEC must be worn to handle any used and contaminated instruments.
- Disinfect and clean instruments immediately after use.

Generally, cleaning in a washer-disinfector unit is preferable to manual cleaning.

#### Pre-treatment

- Carefully remove large unhardened residues from the instruments using a lint-free cellulose tissue. For preliminary disinfection, place the instruments in a disinfection bath (with a lid and sieve insert) filled with aldehyde-free alkaline disinfectant (e.g. immersion in ID 212 forte, Dürr Dental, for 15 min.) Make sure that the instruments are completely covered in disinfectant and that the disinfectant is free of bubbles.
- Only use a soft brush for the manual removal of residues. Never use a metal brush or steel wool.
- For automated cleaning, carefully rinse the instruments under running tap water after preliminary disinfection.

#### Automated cleaning and disinfection

- RDG (e.g. thermal disinfector G7882, Miele)
  - Cleaning and disinfection program (e.g. Vario TD)
  - Cleaning and disinfection agents (e.g. Neodisher FA, Dr. Weigert)
- If the instruments are exclusively machine cleaned, subsequent thermal disinfection in the steam sterilizer is mandatory.

#### Manual cleaning and disinfection

- Soft brush (e.g. instrument cleaning brush with nylon bristles, Mitex)
- Cleaning and disinfection agents (e.g. immersion in ID 212 forte, Dürr Dental, for at least 15 min.)

After cleaning and disinfection, thoroughly rinse the instruments under tap water in the sieve insert.

#### Drying

- Compressed air or clean, lint-free cellulose tissue

#### Sterilization

- Suitable sterile goods packaging (e.g. aluminium sterilization container)
- Autoclave (e.g. Systec HX-320)
- Program: Steam sterilization in a fractionated pre-vacuum process at 134 °C (273 °F) for at least 4 min.

#### Maintenance, examination and inspection

After cleaning and disinfecting the instruments, visually inspect them to make sure that they are clean, undamaged and functioning properly. If macroscopically visible residual debris is present, repeat the cleaning and disinfection process.

#### Storage and shelf life

- Store the instruments in a dry place to avoid condensation build-up.
- Store OptraSculpt and OptraSculpt Pad attachments in the original packaging at room temperature, protected from dust, moisture, compression and contamination.
- Store at 2–28 °C / 36–82 °F
- Expiry date: see printed inscription on label.

#### Notes

-  The hand-held instrument is autoclavable and suitable for multiple use and designed to withstand at least 200 autoclaving cycles. See "Disinfection, cleaning and sterilization" for the stipulated procedures.
- Before autoclaving the hand-held instrument, remove the modelling attachments.
- Forceful removal of the attachments or mechanical manipulation of the holding ring may lead to instrument breakage and should therefore be avoided.
-  The OptraSculpt modelling attachments are intended for single use only. They must not be cleaned, disinfected, sterilized and reused. Cleaning and disinfection agents may have a detrimental effect on the material. Sterilization in the autoclave will destroy the material.
- The use of a rubber dam is recommended in order to avoid accidental ingestion or aspiration of small parts.
-  The Instructions for Use are available on the website of Ivoclar Vivadent AG ([www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)).

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**ivoclar  
vivadent®**  
*clinical*

## Keep out of the reach of children! For use in dentistry only!

The product has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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**Ivoclar Vivadent Ltd.**
**Ivoclar Vivadent Marketing Ltd.**
**Ivoclar Vivadent AB**
**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**