Zahnformenkarte Mould chart

Kunststoffzähne
Resin teeth
Ivostar®/Gnathostar®

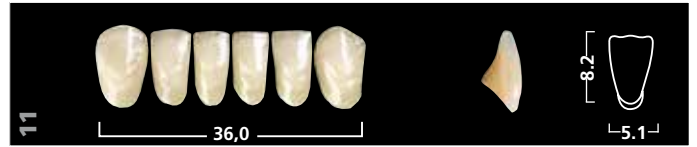
Frontzähne

Ivostar®

obere
uppers

untere
lowers

Seitenzähne

Gnathostar®

Formen

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Moulds

A wide range of 15 upper and 6 lower Ivostar anterior molds; 5 upper and 5 lower Gnathostar posterior molds. The anteriors come in three shapes: triangular, rectangular, and oval – arranged from small to large.

Shades

The Chromascope shade guide comprises 20 shades based on the tried and tested Ivoclar Vivadent dentin shades. The Chromascope is therefore the shade guide for all prosthetic restorations for Ivoclar Vivadent metal-free ceramics, for IPS Empress, as well as for partial and complete dentures. Also moulds are available in 16 A–D and 4 Bleach shades.

Layering

Harmonized layering gives Ivostar/Gnathostar teeth the natural appearance of the original tooth.

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Criteria for Selection

There are numerous methods employed in the selection of teeth, based on such criteria as the shape of the head, bodily structure, and male or female characteristics. Even if no information is available on the size, shape, and position of the teeth, valuable points of reference can be obtained from the form of the maxilla. From the vast range of maxillary forms identifiable, three basic types can be inferred:

- square
- triangular
- oval

If these three forms are used to group Ivoclar Vivadent teeth according to their anatomic characteristics, the following classification results.

The selection of the lower Ivoclar Vivadent anteriors is done with help of the combination chart (mould cart).

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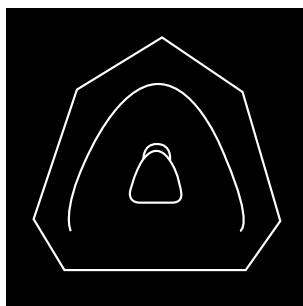
Tooth shades are chromatically divided into 5 groups and numerically organized according to shade intensity.

The Chromoscop shade guide is the uniform shade standard for the entire Ivoclar Vivadent product range.

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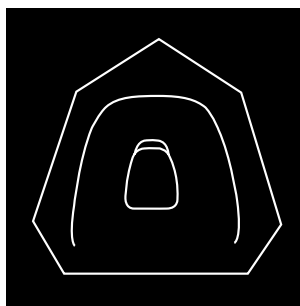
31, 32, 33, 34, 35.

These groups are characterized by teeth having a rejuvenating crown which tapers towards the cervical ridge.

Gingivo-incisal curvature is somewhat more pronounced. The large central incisor has an attractive, slightly triangular form.

The residual ridge is also triangular and basically indicated for cross-bite.

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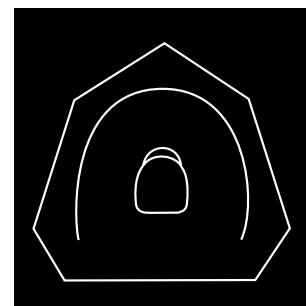
41, 42, 43, 44, 45.

These molds are basically quadrilateral and the central incisor relatively dominant.

Gingivo-incisal curvature is moderate. The residual ridge is markedly square and tends to be indicated for cases involving normal bite.

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01, 02, 03, 04, 05.

These groups are anatomically characterized by pronounced gingivo-incisal curvature. An oval residual ridge is often found in cases of deep overbite.

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Shades: Ivostar/Gnathostar teeth are selected and coordinated with the Chromoscop and/or Ivostar/ Gnathostar shade guide.

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© Ivoclar Vivadent AG, Schaan/Liechtenstein
Ivoclar Vivadent AG
Ivoclar Vivadent GmbH