casebook

The future of
composite technology

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One layer, one impressive result. Everyone working with Tetric EvoCeram® Bulk Fill knows about the advantages of this posterior composite. Three patented key components provide the decisive advantages in modern bulk-fill technology: the Ivocerin® light initiator, the light controller and the shrinkage stress reliever. All experts agree on this.

In this Case Book internationally renowned dentists demonstrate how they have achieved excellent results with Tetric EvoCeram Bulk Fill.

Dr. David Hacmoun

Privatpraxis in Antibes, Frankreich
Private practice in Antibes, France

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To minimize the shrinkage stress, composites should be applied in thin layers and intermediately cured. Tetric EvoCeram Bulk Fill simplifies this application procedure for direct composite restorations, particularly for restorations in the posterior region.

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Starting situation:
Caries on teeth 16 and 17.

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The cavity was filled with a composite layer of shade IVW and the fissures were subsequently characterized with IPS Empress Direct® Color.

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The carious dentin near the pulpa remained. The final excavation was carefully performed under the microscope using an excavator.

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Etching the enamel and dentin with Total Etch.

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View of the etched tooth structure. The enamel has the typical chalky-whitish colour.

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After Excite® F had been applied and polymerized, the discoloured dentin of tooth 16 was masked with Tetric EvoFlow®, shade A1.

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Build-up of the proximal walls with a layer of Tetric EvoCeram® Bulk Fill, shade IVW.

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The high translucency has so far been a severe limitation in the use of bulk-fill composites. The restorations had a greyish appearance and esthetics were insufficient. Tetric EvoCeram Bulk Fill changed all this. Due to the low translucency, no compromises in terms of esthetics have to be made anymore. Furthermore, the time-consuming layering procedure is rendered redundant with Tetric EvoCeram Bulk Fill.

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Starting situation: mesial and occlusal caries on tooth 36.

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Situation after placing a sectional matrix, fixed with a wooden wedge. The matrix fits snugly so that no metal ring is required.

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Both cavities were filled with a layer of Tetric EvoCeram Bulk Fill, shade IVB. The photo shows the situation before polishing.

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View after completed treatment.

Dr. Markus Lenhard

Privatpraxis in Neunkirch, Schweiz
Private practice in Neunkirch, Switzerland

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I believe that Tetric EvoCeram Bulk Fill is a very good composite material with excellent handling properties. Furthermore, it simplifies and expedites the layering procedure. You can almost say that, once you have worked with Tetric EvoCeram Bulk Fill, you no longer want to work with conventional composites.

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Starting situation: Fracture of the lingual wall of tooth 36, buccal cusps with fractures.

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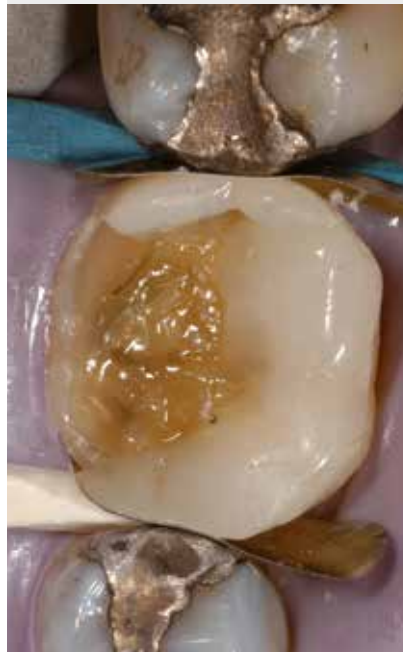
First increment: the acceptable layer thickness was not fully exhausted in order to correctly form the marginal ridge first.

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Recall after one year.
The clinical situation is unchanged.

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I have not found any reason so far to return to conventional nanohybrid composites. Since I have started using Tetric EvoCeram Bulk Fill, I have changed my concept when performing posterior restorations. Even for those who are not yet fully convinced of the bulk-fill technique: The material can also be used with the conventional technique with the advantage of a reduced shrinkage rate and a greater curing depth.

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The result one week after the restoration: an optimum reconstruction of the lower anterior region.

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Placing the retraction cord and removing caries and stains; the preparation margins are clearly discernible.

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To preserve the gingiva, a retraction cord was placed before etching, bonding (Syntac®) and placing the composite.

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The result one week after the
restoration: an optimum
reconstruction of the lower
anterior region.

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Department of Operative Dentistry and Periodontology
Munich, Germany

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With the bulk-fill composite Tetric EvoCeram Bulk Fill I can effectively restore even large posterior cavities while obtaining maximum quality. At the moment, it can be assumed that Tetric EvoCeram Bulk Fill will show a low annual loss rate similar to that of other proven micro- and nanohybrid composites (e.g. Tetric EvoCeram).

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Starting situation: old, fractured composite filling in tooth 16 and mesial caries.

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Final situation: Tooth shape and esthetics have been successfully reconstructed.

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After conditioning of the cavity with ExcīTE® F, Tetric EvoCeram Bulk Fill was applied.

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The first layer was so thin that an additional composite layer with a maximum height of 4 mm could be placed in a next step.

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With the second layer, the remaining volume of the cavity was filled.

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Shaping a functional but rational occlusal anatomy.

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Light-curing of the second layer for 10 seconds with the Bluephase® Style curing light.

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Tetric EvoCeram Bulk Fill gives me more freedom in restorative therapy. I can work more efficiently and adapt the layering process to the clinical situation.

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Restoration at recall
after one week.

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Filling the defect with
Tetric EvoCeram Bulk Fill.

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Restoration at recall after
one week.

Marcos Vargas, DDS, MS

Professor, Abteilung für Familienzahnheilkunde
The University of Iowa College of Dentistry
Iowa City, USA
Professor, Department of Family Dentistry
The University of Iowa College of Dentistry
Iowa City, USA

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The cumbersome, conventional incremental filling technique is rendered entirely redundant with Tetric EvoCeram Bulk Fill. Therefore, the dentist can now place permanent and esthetic posterior restorations in half the time previously needed. I could reduce my treatment time for Class II restorations.

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Starting situation: Permeable amalgam filling in tooth 26 of the maxilla.

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After isolation of the treatment area, the amalgam filling and the carious tissue was removed. A sectional matrix was placed in order to obtain close proximal contacts.

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The adhesive was applied after etching.

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The cavity was filled with a single layer of Tetric EvoCeram Bulk Fill.

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Anatomic contouring with subsequent light-curing using the Bluephase® Style curing light.

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Tetric EvoCeram Bulk Fill is easily applied and allows for reliable and reproducible results in direct posterior restorations. Its easy mouldability, its curing depth of 4 mm and its long processing and modelling time under operatory light enable a fast restoration of even large cavities. The material can be optimally adapted in the occlusal and cervical area.

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Starting situation: Caries in the occlusal region of tooth 24, 25.

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Excavation of the carious tissue while protecting the adjacent tooth.

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Isolation of the surgical area using a rubber dam.

4

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Sectional matrix fixed with V3 Ring – Wam.

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Tetric EvoCeram Bulk Fill in shade IVA was used for the restoration.

Dr. Rafael Piñeiro

Privatpraxis in Cambados, Spanien
Private practice in Cambados, Spain

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The esthetic results which can be obtained easily and quickly with Tetric EvoCeram Bulk Fill make it my material of choice.

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Starting situation: Occlusal caries
on tooth 36 and 37.

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Situation after removal of the carious tooth structure; measuring the depth of the cavity.

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After polymerization of the adhesive, Tetric EvoCeram Bulk Fill was applied.

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Ivoclar Vivadent AG