Description

Adhese® Universal is a light-curing single-component dental adhesive for enamel and dentin and is compatible with all etching techniques (self-etch, selective-enamel-etch and etch & rinse techniques).

Composition

Adhese Universal contains methacrylates, ethanol, water, highly dispersed silicon dioxide, initiators and stabilizers.

Indications

- a. Indications when light-cured at 500 to 1,400 mW/cm² or 1,800 to 2,200 mW/cm²**
- Direct-placed light-curing composite and compomer restorations
 - Direct-placed core build-ups with light-, self- and dual-curing composites
 - Repair of fractured composite and compomer restorations
 - Adhesive cementation of indirect restorations with light- and dualcuring luting composites
 - Sealing of prepared tooth surfaces before temporary / permanent cementation of indirect restorations
 - Desensitization of hypersensitive cervical areas (Note: Provide additional cooling with an air syringe when applying the product).
- b. Indications when light-cured using Bluephase® PowerCure in the 3sCure curing mode**
- Restorations in the posterior region of permanent dentition (Classes I and II, including the replacement of individual cusps) when light-cured from the occlusal aspect

Safety notes

- Do not place light in direct contact with unprotected gingiva, mucous membrane or skin.
- The 3sCure mode must not be used in case of caries profunda and very deep cavities.

Contraindications

- Do not use Adhese Universal if the patient is known to be allergic to any of the materials' ingredients or if the stipulated working technique cannot be employed.
- Applications in which sufficient illumination cannot be ensured (e.g. cementation of root canal posts).
- Direct pulp cappings.

Side effects

In rare cases, components of Adhese Universal may lead to sensitization. In such cases, Adhese Universal should no longer be used.

Interactions

- Peroxide-based bleaching agents and disinfectants may inhibit the polymerization of adhesives. Such materials must not be used ≤ 14 days prior to the application of Adhese Universal.

Adhese® Universal

EN	Instructions for Use Light-curing single-component dental adhesive	FI	Käyttöohjeet Valokovetteinen yksikomponenttinen sidosaine
DE	Gebrauchsinformation Lichthärtendes Einkomponenten-Dentaladhäsiv	NO	Bruksanvisning Lysherdende enkomponent-dentaladhäsiv
FR	Mode d'emploi Adhésif dentaire monocomposant photopolymérisable	NL	Gebruiksaanwijzing Lichtuithardend ééncomponentadhesief voor tandheelkundig gebruik
IT	Istruzioni d'uso Adesivo dentale monocomponente	EL	Οδηγίες Χρήσεως Φωτοπολυμερίζόμενο οδοντιατρικό συγκολλητικό ενός φασιδίου
ES	Instrucciones de uso Adhesivo dental mono-componente y fotopolimerizable	TR	Kullanma Talimatı Işıkla sertleşen tek bileşenli dental adeziv
PT	Instruções de Uso Adesivo dental fotopolimerizável monocomponente	RU	Инструкция по применению Светоотверждаемый однокомпонентный стоматологический адгезив
SV	Bruksanvisning Ljushärdande en-komponents dentaladhäsiv		

- Phenolic substances (e.g. eugenol) may inhibit the polymerization of Adhese Universal.
- Before any treatment with Adhese Universal, completely rinse off all iron- and aluminium-based styptic agents with water.
- Fluoridation agents must not be used prior to the application of Adhese Universal.

Procedure for direct and indirect restorations

1. Isolation

Adequate relative or absolute isolation using suitable auxiliaries, such as OpraGate® or OpraDam® Plus is required.

2. Pulp protection / cavity liner

The 3sCure curing mode must not be used for restorations in areas close to the pulp. In very deep cavities, areas close to the pulp must be selectively coated with a calcium hydroxide liner (e.g. ApexCal®); and subsequently covered with pressure-resistant cement (e.g. a glass ionomer cement).

3. Conditioning with phosphoric acid gel (optional)

The bond to enamel can be improved by selectively etching the enamel or by applying the "etch & rinse" technique. Unprepared enamel surfaces must be conditioned with phosphoric acid (e.g. Total Etch). Please observe the instructions for use for the phosphoric acid gel.

a. Selective-enamel-etch technique

Apply phosphoric acid gel (e.g. Total Etch) onto the enamel and allow it to react for 15–30 seconds. Then rinse thoroughly with a vigorous stream of water for at least 5 seconds and dry with oil- and water-free compressed air until the etched enamel surfaces appear chalky white.

b. Etch & rinse technique

Apply phosphoric acid gel (e.g. Total Etch) onto the prepared enamel first, and then onto the dentin. The etchant should be left to react on the enamel for 15–30 seconds and on the dentin for 10–15 seconds. Then rinse thoroughly with a vigorous stream of water for at least 5 seconds and dry with oil- and water-free compressed air until the etched enamel surfaces appear chalky white.

4. Handling VivaPen and bottle

a. VivaPen

- Remove the VivaPen cap by turning it until a noticeable resistance is overcome whilst simultaneously pulling it off.
- Attach the VivaPen brush cannula. The brush cannula can be rotated up to approximately 270° to the desired position.

Important: Rotating the cannula past the point of resistance will detach the cannula!

- For hygienic reasons, always use a VivaPen protective sleeve by sliding the Pen with the cannula into the protective sleeve until the cannula punctures through the sleeve.
- A new cannula and protective sleeve must be used for each patient.
- Activate the click mechanism 2–3 times until the cannula tip is sufficiently coated with Adhese Universal. The wetting of the cannula tip is recognized by the yellow colour of Adhese Universal.
- If used several times on the same patient, the moistened cannula must be protected from light.
- Remove the protective sleeve and cannula after having used the VivaPen. The VivaPen brush cannula is removed by turning it beyond the point of resistance and simultaneously pulling it off.
- Close the VivaPen with the original cap.

Notes

- When using the VivaPen for the first time, more than 2–3 clicks are required to wet the cannula rush due to the engineering of the VivaPen. This does not influence the indicated fill-level or the function of the VivaPen.
- Do not immerse the VivaPen in disinfectant solutions or autoclave it.
- Contamination on the VivaPen can be removed, e.g. by wiping with alcohol or other non-oxidizing disinfectants.
- For hygienic reasons, VivaPen protective sleeves should always be used.
- Cannulas other than those supplied with the VivaPen are not compatible with the VivaPen and must not be used.
- The fill-level indicator of the VivaPen identifies the remaining amount of adhesive: if the indicator is no longer visible on the display, the VivaPen is empty.

b. Bottle

- Dispense the desired amount of Adhese Universal into a mixing well and apply it using a disposable applicator (e.g. Vivadent® Applicator Brush).
- Protect the adhesive from light (e.g. VivaPad®).
- Carefully close the bottle immediately after each use.
- A new disposable applicator must be used for each application.

Notes

- Contamination on the bottle can be removed, e.g. by wiping with alcohol or other non-oxidizing disinfectants.
- Do not use the bottle intraorally. For hygienic reasons, suitable application aids should be used (e.g. VivaPad®).

5. Application of the adhesive

- Starting with the enamel, completely coat the tooth surfaces to be treated with Adhese Universal.
- The adhesive must be scrubbed into the tooth surface for at least 20 seconds. This time must not be shortened. Applying the adhesive on the tooth surface without scrubbing is inadequate.
- Disperse Adhese Universal with oil- and moisture-free compressed air until a glossy, immobile film layer results.

Important note: Avoid pooling, since this can compromise the fitting accuracy of the permanent restoration.

6. Light-curing the adhesive

Adhese Universal can either be light-cured for 10 s at a light intensity of 500 to 1,400 mW/cm² or for 5 s at a light intensity of 1,800 to 2,200 mW/cm². In the fabrication of Class I and II restorations, Adhese Universal can be light-cured in only 3 s from the occlusal aspect at a light intensity of 2,700 to 3,300 mW/cm² (e.g. using Bluephase PowerCure).

The instructions for use of the curing light used must be observed.

light intensity (mW/cm ²)	exposure time
500–900	10 s
900–1,400	10 s
1,800–2,200	5 s
2,700–3,300	3 s

7. Application of the restorative or luting composite

- Proceed according to the instructions for use of the restorative and/or luting composite used.

Procedure for intraoral repairs of fractured composite fillings/composite restorations with light-curing materials

- Roughen the surface to be repaired using diamond finishers and thoroughly clean with water spray and dry with oil- and water-free compressed air.
- Proceed according to the direct/indirect application for Adhese Universal.

Procedure for the conditioning of Tetric® CAD restorations

- Proceed according to the instructions for use for Tetric CAD.

Warnings for Adhese Universal

- Adhese Universal is irritant. Avoid contact with the skin, mucous membrane and eyes.
- In case of skin contact, immediately wash with soap and copious amounts of water.
- If the material should accidentally come in contact with the eyes, immediately rinse with copious amounts of water and consult a physician / ophthalmologist.
- In rare cases, contact with the skin may lead to sensitization towards the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effects of methacrylate.
- Do not use Adhese Universal as a primer for ceramic restorative materials. A suitable ceramic primer must be used (e.g. Monobond Etch & Prime®).

Warnings for phosphoric etching gel

Phosphoric etching gel is corrosive. Avoid contact with the eyes, mucous membrane and skin (protective goggles are recommended for both the patients and practitioners). If the material should accidentally come in contact with the eyes, immediately rinse with copious amounts of water and consult a physician / ophthalmologist.

Shelf life and storage

- Expiration date: see imprint on packaging.
- Storage temperature: 2–28 °C / 36–82 °F.

Keep material out of the reach of children!

For use in dentistry only!

The product has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

The Material Safety Data Sheet is available online at www.ivoclarvivadent.com

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Ivoclar Vivadent Marketing Ltd.
Ivoclar Vivadent AB
Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.