


Tetric EvoFlow®

1


Self-Etch



Etch & Rinse

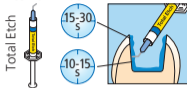


Selective Enamel-Etch

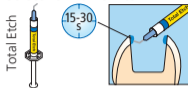

2

Adhese® Universal
VivaPen®

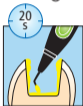

Bluephase®
PowerCure

3

Tetric®
EvoFlow®




Tetric®
PowerFlow



4

Tetric® Prime

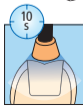

or







Tetric® PowerFill

mW/cm ²	Tetric Prime/ Tetric EvoFlow	Tetric PowerFill/ Tetric PowerFlow	Adhese Universal
500 – 900	20 s	20 s	10 s
900 – 1400	10 s	10 s	10 s
1800 – 2200	5 s	5 s	5 s
Direct posterior restoration, light-cured from occlusal aspect Direkte Seitenzahnrestauration, okklusale Belichtung			
2700 – 3300	–	3 s	3 s

ivoclar
vivadent
clinical

Tetric EvoFlow®

	Conventional composites				4-mm-composites	
	FLOWABLE		SCULPTABLE		FLOWABLE	SCULPTABLE
	Tetric EvoFlow®		Tetric® Prime		Tetric® PowerFlow	Tetric® PowerFill
						
	Dentin & Bleach XL	Enamel	Dentin	Enamel	Dentin-like translucency	Enamel-like translucency
Layer thickness up to	1.5 mm	2 mm	1.5 mm	2 mm	4 mm	4 mm
Exposure Time ≥ 500 mW/cm²	20 s				20 s	
900 – 1 400 mW/cm²	10 s				10 s	
1 800 – 2 200 mW/cm²	5 s				5 s	
2 700 – 3 300 mW/cm², e.g. Bluephase® PowerCure	–				3 s	

3s

CE 0123

ivoclar
vivadent
clinical

Date information prepared: 2019-12-10/64 1319/Rev. 4
Ivoclar Vivadent AG, 9494 Schaan/Liechtenstein



see
instructions