# Impressive esthetics Beeindruckende Ästhetik

**Dr Gabriel Krastl**  
Switzerland

**Dr Andrea Shepperson**  
New Zealand

**Dr Martin von Sontagh**  
Austria

**Marshall Hanson**  
USA

**Dr Jorge Eustáquio**  
Brazil

**Jan Kurtz-Hoffmann**  
Germany

**Dr Eduardo Mahn**  
Chile

**Dr Markus Lenhard**  
Switzerland

**Prof. Dr Antonio Cerutti**  
Italy

**Dr Olivier Romieu**  
France

**Dr Rafael Piñeiro Sande**  
Spain

**Dr Tomasz Sosnowski**  
Poland

**IPS Empress® Direct**  
Casebook

**ivoclar**  
**vivadent®**  
passion vision innovation

The universal nano-hybrid composite IPS Empress® Direct is synonymous with utmost esthetics in the restoration of dental defects. In addition to providing lifelike opacity and fluorescence, IPS Empress Direct impresses both scientists and users with its true-to-nature shade scheme.

This casebook gathers together the different methods employed by internationally acclaimed dentists when restoring teeth with IPS Empress Direct and shows the truly exceptional results they have been able to achieve with this material.

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# Dr Gabriel Krastl

“

IPS Empress® Direct provides everything you need for modern, highly esthetic composite dentistry: a wide range of materials with differing levels of opacity and outstanding stains for the individual characterization of the restorations. What is more, the system is forgiving of minor layering errors and thus enables predictable esthetic results to be achieved.

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### Case presentation

In the case shown, an anterior tooth was reconstructed with the aid of a silicone key. First, the artificial palatal "enamel shell" was built up. This was followed by the reconstruction of the dentin core and the creation of mamelon structures. Sufficient space was left for the subsequent layering of translucent material. The composite build-up was completed with a thin, translucent enamel layer. In the polishing step, the surface characteristics of the natural dentition were reproduced in the restoration.

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after | nachher

Starting situation: Crown fracture of tooth 21

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A rubber dam is placed with ligatures around teeth 11 and 21.

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The palatal wall is rebuilt with IPS Empress Direct Enamel A1 using the silicone key as a guide.

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The completed palatal and proximal walls

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IPS Empress Direct Dentin A2 is used to reconstruct the dentin core and create mamelon structures.

Dentinkern aus IPS Empress Direct Dentin A2 schichten und Mamelonstrukturen modellieren

The dentin core and the incisal edge are characterized with white stains (IPS Empress Direct Color).

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Translucent material (IPS Empress Direct Opal) is applied in the incisal region.

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IPS Empress Direct Enamel A1 as the final covering layer completes the build-up.

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Situation immediately after polishing

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Postoperative view showing the seamless integration of the composite resin restoration

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# Dr Eduardo Mahn

“

I have been using IPS Empress® Direct for quite some time now. It's the simplicity of the layering procedure, the lifelike shades and long-lasting lustre that won me over. The wide array of enamel and dentin materials available covers any tooth shade a patient may exhibit. And if that's not enough, the shades can easily be combined. To me this material is simply ideal!

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### Case presentation

Due to the incorporation of nano-fillers and improvement of the optical properties, composite restoratives are now almost as esthetic as modern ceramics.

When the treatment plan for this case was established, tooth proportions, hue, value, colour saturation and the gingival contour were defined in line with the pink esthetics and facial features of the patient. This report describes how we succeeded in profoundly changing the smile of the 24-year-old patient with the help of IPS Empress Direct.

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after | nachher

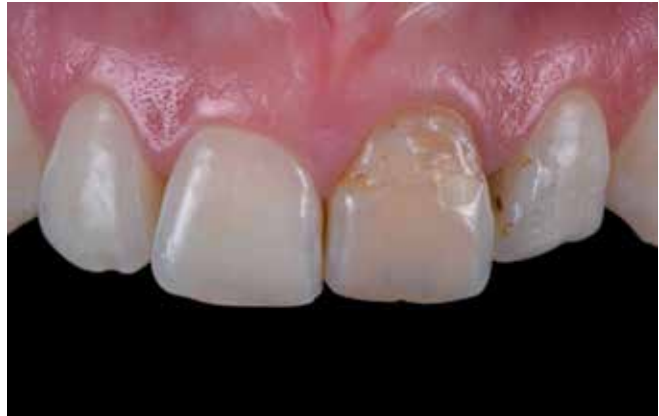
Starting situation: Defective composite filling in tooth 21

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
Treatment planning

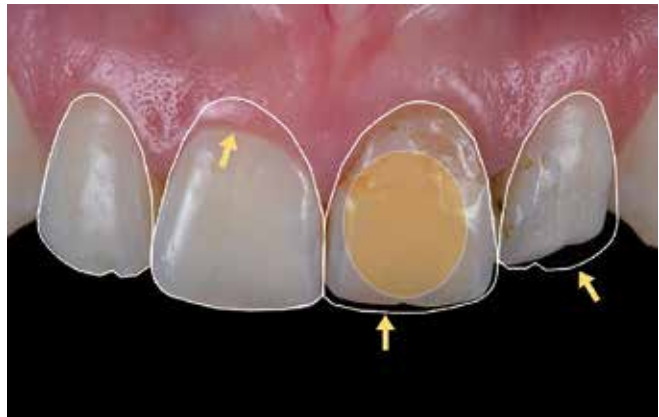Planung

Bleaching of the central incisor in the dental practice using 38% hydrogen peroxide (for 20 minutes)

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IPS Empress Direct is used to create a direct resin mock-up in order to check the appropriate length and tooth proportions.

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The silicone key, which is based on the mock-up, serves as a guide during the entire composite build-up procedure.

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IPS Empress Direct Dentin Bleach L and Enamel Bleach L are placed and adapted with the new composite modelling instrument OptraSculpt® Pad.

IPS Empress Direct Dentin Bleach L und Enamel Bleach L mithilfe des neuen Composite-Modellier-instruments OptraSculpt® Pad adaptieren

Crown lengthening is performed using oscillating instruments (Komet, Brasseler) in order to adjust the gingival outline of teeth 11 and 12.

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The restoration surfaces are finished with burs to create a natural-looking surface texture.

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Lateral view: Perfect blend of IPS Empress Direct with the natural tooth structure

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An outstanding result is achieved due to the excellent optical properties of IPS Empress Direct and the clear identification of the esthetic challenges prior to the start of the treatment.

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Patient beaming with self-confidence after completion of the treatment

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# Marshall Hanson, DDS

A handwritten signature in black ink, appearing to read 'Marshall Hanson'. The signature is fluid and cursive, with a long horizontal line extending to the right.

“

When restoring front teeth, especially when adding back length, IPS Empress® Direct has consistently given me predictably esthetic results. In particular I like the opacity and translucency of the dentin and enamel shades that allow me to create natural-looking restorations. The handling properties and polishability of IPS Empress Direct are exceptional.

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### Case presentation

The effect that a restored and enhanced smile can have on a patient is indescribable. Providing this type of service for someone is really the root of what makes us passionate about the pursuit of excellence in esthetic dentistry. Having treatment options where we can achieve high levels of esthetics responsibly through minimally to non-invasive procedures, like direct resin veneers with IPS Empress Direct, truly makes us all smile a little broader and brighter. This is corroborated by the case below.

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after | nachher

Starting situation: Fracture of teeth 11 and 21

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Retracted view of the preoperative situation

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Preparation prior to etching and bonding

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Tooth 11: B1 Dentin shade is selected to build up the lingual-incisal portion, and B1 Enamel shade is placed and sculpted on the facial aspect.

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Tooth 11 is shaped and refined with discs and a fine diamond bur.

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Teeth 14 to 24 are each built up in the same fashion as tooth 11, using A1 Dentin and Enamel shades for the laterals and bicuspid and A2 for the cuspids. A cut-back is then done to create room for incisal characterization and translucency.

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Teeth 13 to 23: Dentin IVA6 and Dentin Bleach XL are applied for characterization.

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Trans Opal is then placed for translucency.

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Retracted view post-treatment: The restorations have been polished with rubber points, fine discs and composite polishing paste.

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# Dr Jorge Eustáquio

“

IPS Empress® Direct impresses me with its optimal processing properties, excellent ability to mimic natural tooth structure, natural opacity and superior lustre and polishing properties. A material that is simply amazing!

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### Case presentation

The treatment recommended in the present case included a tooth whitening procedure as well as the restoration of teeth 11, 21 and 22 with composite resin. Wax-ups of the final restorations were created on the gypsum model. After having performed the bleaching procedure, the old restorations were removed and the teeth were restored.

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after | nachher

Starting situation: Esthetic defects in teeth 11, 21 and 22

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Preparation

Präparation

The dentin core is reconstructed with IPS Empress Direct Dentin A2 and Dentin A1. For the palatal enamel portion, IPS Empress Direct Enamel A1 is used.

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IPS Empress Direct Enamel A1 and Trans Opal (11) are applied on the vestibular aspect and IPS Empress Direct Trans 30 is used to create incisal effects.

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Polishing is performed with OpraPol®.

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The completed restorations in an occluded position

Fertige Arbeit – in Okklusion

The completed restorations – lateral view

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The completed restorations in the smile

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Display of the teeth with the lips in repose

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# Jan Kurtz-Hoffmann

“

With IPS Empress® Direct we have a highly esthetic filling material at our fingertips. Its broad range of materials and shades provides exceptional flexibility in the layering of composite resin. As a result, restorative procedures can be tailored to the individual situation.

IPS Empress Direct facilitates day-to-day practice routines as it is convenient to apply, does not stick to the instrument and can be easily polished to a high lustre. Anatomical layering of the composite resin allows predictable treatment results to be achieved. Even complex reconstructions in combination with indirect restorations are possible, with impressive esthetic results.

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### Case presentation

A 49-year-old patient presented in our office desiring an improvement of the esthetic appearance of her upper teeth. The defective fillings in teeth 11 and 12 bothered her and she wanted more than just having the low-grade cervical defects along the gingival margin in the upper anterior region and the point of the canine restored. After receiving a comprehensive patient consultation, which included a wax-up and mock-up, she decided to have minimally invasive treatment: this included a bleaching procedure and a correction of the fillings in teeth 11 and 12 using a direct restorative approach. After having placed a rubber dam, the teeth were reconstructed with IPS Empress Direct composite using a silicone key produced from the wax-up on the plaster model.

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after | nachher

Starting situation: Defective fillings with secondary caries in teeth 11 and 12

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Conditioning of the enamel and dentin:  
The enamel is etched for 3 s, the dentin for 15 s.  
Then Syntac® adhesive is applied.

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The palatal wall is rebuilt with the help of the silicone key.

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For this purpose, IPS Empress Direct Enamel A1 is used.

... und IPS Empress Direct Enamel A1

The mesial wall is shaped with IPS Empress Direct Enamel A1 composite. For the core, Dentin A1/A2 is used.

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IPS Empress Direct Color blue is used to add translucency.

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Build-up of the vestibular wall using IPS Empress Direct Enamel A1

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After having built tooth 12 to its final form, it is finished and pre-polished.

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Build-up of the palatal wall using IPS Empress Direct Enamel A1

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The distal wall is rebuilt using IPS Empress Direct Enamel A1. For the core, Dentin A1/A2 is used.

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Olivier Romieu

“

IPS Empress® Direct is an exceptional product which enables the realization of highly esthetic anterior restorations. It allows the tooth morphology, gloss and surface characteristics as well as all the optical effects of natural teeth to be accurately reproduced. The shade guide and flowable effect shades significantly facilitate the fabrication of restorations.

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”

### Case presentation

A 24-year-old patient wanted to have her upper central incisor restored. Due to an old composite resin restoration, the tooth exhibited two different colours. At the age of nine, the patient had suffered trauma to this anterior tooth. Endodontic treatment was performed and the tooth restored with composite resin.

Following the examination of the remaining tissue, it was decided to choose a conservative treatment approach including internal bleaching.

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after | nachher

Starting situation: Tooth 21 showing two different colours due to an unsightly composite resin restoration and discolouration of the remaining tissue

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Tooth 21 after two weeks of internal bleaching with sodium perborate

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After the etching procedure, Excite® F adhesive is applied with the VivaPen®.

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Following the placement of a metal matrix,  
the mesial and palatal walls are rebuilt with  
IPS Empress Direct.

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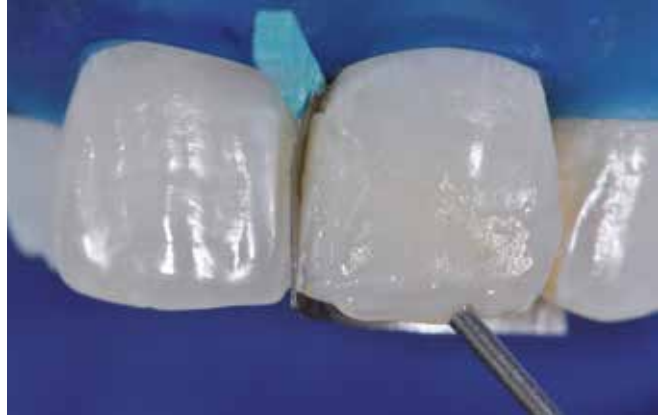
Modelling with OptraSculpt® Pad

Modellieren mit OptraSculpt® Pad

IPS Empress Direct Flow Trans Opal is applied.

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Two weeks postoperatively – teeth in function

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The patient flashes a carefree smile again.

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# Dr Martin von Sontagh

“

I primarily use IPS Empress® Direct for esthetic and natural-looking resin restorations. Due to the wide array of shades available, teeth can be rebuilt layer by layer in a lifelike fashion. Moreover, IPS Empress Direct is particularly suitable for bruxing patients because of its high wear resistance.

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### Case presentation

Many people process stress by grinding their teeth at night. Therefore, bruxism is a very frequently diagnosed disorder. Restorations with state-of-the-art composite resin such as IPS Empress Direct are often the treatment of choice in such cases, as they enable a minimally invasive restorative approach. Even patients with functional problems can be treated conservatively.

In the present case the patient wished the gap between his front teeth to be closed and the tissue defects to be restored. However, he declined orthodontic treatment. After having analyzed the photos and the model, a personalized treatment plan was designed. The tooth length and width proportions were newly defined and the axes of teeth 14 to 24 were reoriented by means of composite resin veneers.

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after | nachher

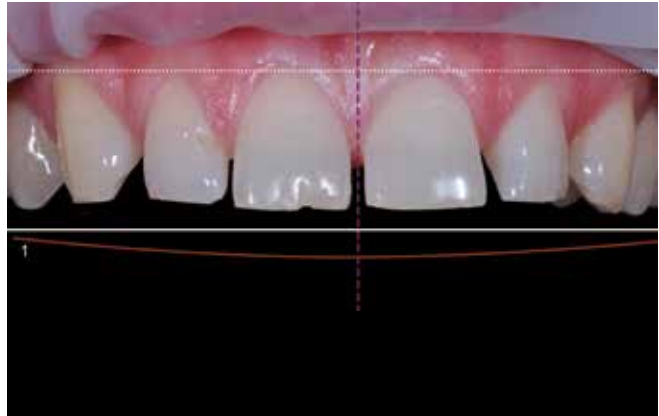
Starting situation: The functional problems and diastemata are obvious.

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A photographic analysis shows that the incisal line does not correspond with the natural smile line (1).

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The cranial view confirms that the tooth axes are not straight and the edges of the teeth do not follow the curve of the lip.

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Delicate veneer preparations are made to align the teeth along one axis.

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Then the teeth are built up in layers using different shades of IPS Empress Direct.

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The composite resin is shaped and adapted with OptraSculpt® Pad until a smooth surface is achieved.

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The reference scales on the OptraSculpt Pad handle assist in determining the tooth width.

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Result after completion of the treatment

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A realignment of the incisal line with the smile line has been achieved, which ensures a more natural look.

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*Tomasz Sosnowski*

“

IPS Empress® Direct offers clinicians a range of materials suitable for customized layering, particularly in the incisal region. Mimicking the shape and shade of natural teeth as well as reproducing individual tooth characteristics is particularly easy with this restorative system. IPS Empress Direct allows me to restore the natural anatomy and appearance of teeth in a targeted fashion, especially along the incisal edge.

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### Case presentation

A 25-year-old patient wanted to have the defective and unnatural-looking composite restoration in tooth 21 replaced. She did not want to have the tooth prosthetically restored with ceramic material due to the costs involved and expressed her desire to receive minimally invasive treatment.

The old, defective composite restoration was removed. A preparation depth of 0.5 mm was sufficient in this case and the incisal angle of the tooth was included into the restoration.

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after | nachher

Starting situation: unnatural-looking composite restoration in tooth 21

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Close-up view

Nahaufnahme

Situation following the removal of the existing restoration and tooth preparation

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After having etched the preparation with Total Etch, the adhesive is applied.

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The basic shape of the restoration is established with the first increment of Enamel A2 composite.

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Dentin A2 composite is applied and adapted to the natural tooth structure.

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Tooth characteristics are imitated with IPS Empress Direct Color blue, white and honey yellow.

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IPS Empress Direct Trans 30 is placed along the incisal edge to attain a translucency effect.

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The last IPS Empress Direct Enamel A2 layer is shaped with OptraSculpt Pad.

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Fluoride treatment with Fluor Protector S

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Result achieved with minimal removal of natural tooth structure

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# Dr Rafael Piñeiro Sande

“

IPS Empress® Direct offers everything you would expect from an esthetic composite material at a very high level: Fluorescence, translucency, polishability, effect materials and a variety of shades. In addition, the material's physical properties are outstanding. That's why I use IPS Empress Direct.

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”

### Case presentation

A 72-year-old patient presented to my practice with the wish to have his fractured anterior tooth restored. The clinical examination revealed crown fracture of tooth 21 as well as sclerotic alterations in the root area, but no signs of gingival recession or tooth mobility. The X-rays did not show any abnormalities. For a more detailed diagnostic analysis, an impression was taken, a mock-up created and photos were made. The objective of the treatment plan was to create an esthetic composite restoration that faithfully mimicked the appearance of the tooth before the fracture and seamlessly blended into the oral surroundings.

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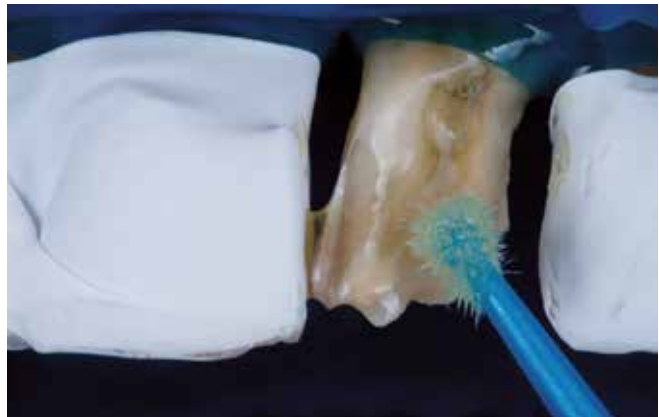
A silicone key is fabricated which is used as a shaping aid for the palatal build-up and to determine the overall dimensions of the restoration.

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The surface is etched with Total Etch for 15 s. Subsequently, Excite® F is applied and rubbed into the prepared tooth structure for 20 s.

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IPS Empress Direct Trans 20 is used to build up the palatal wall.

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A layer of IPS Empress Direct Dentin IVA6 is applied.

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A layer of Enamel A3.5 is placed in such a way that it has uneven thickness and does not cover the vestibular surface completely.

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Prior to light-curing, grooves are cut into the surface with size 06 root canal files. This simplifies the subsequent characterization with IPS Empress Direct Color.

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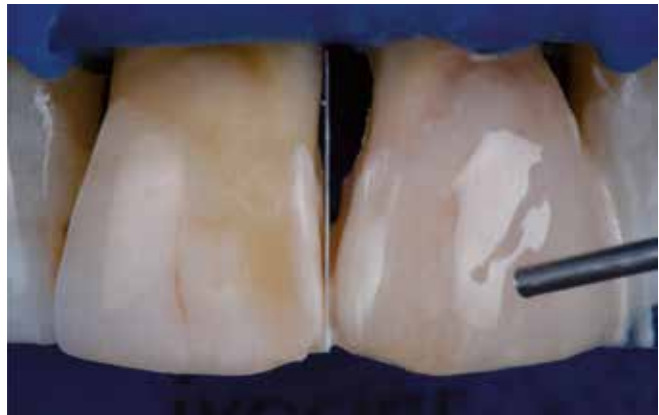
In close proximity to the saturated, white surfaces of the neighbouring teeth, Dentin B1 is placed.

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After the stains have been applied, the vestibular surface is covered with IPS Empress Direct Flow Trans Opal.

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Final situation 15 days after completion of the treatment

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*Andrea Shepperson*

“

IPS Empress® Direct is an intuitive material that morphs with the natural tooth, making it a reliable restorative material for conservative restoration. The shade selection matches a dentist's innate sense of colour perception.

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### Case presentation

The patient presented with a composite repair to her natural tooth, which was placed to mask an area of discolouration on tooth 21. The old restoration was shorter than the adjacent tooth and the incisal edge discrepancy dominating her smile was very obvious. A diagnostic wax-up was constructed and used to fabricate a silicone key to define the incisal edge anatomy. The underlying discolouration was masked with an IPS Empress Direct Dentin shade, chosen for its value and compatibility with tooth 11. An Enamel shade was selected for the body of the tooth and blended short of the facial finish line. A brighter Enamel shade was used to highlight ridges and the incisal edge. Trans 30 completed the facial build-up to obtain an invisible margin. Polishing and contouring was done with Astropol® polishers.

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after | nachher

Starting situation: Incisal edge discrepancy is very visible.

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Marked difference in incisal edge length between teeth 11 and 21

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Careful occlusal adjustment and finishing allows tooth 21 to be lengthened without compromising function.

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Completed restoration in the smile

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# Dr Markus Lenhard

“

Since IPS Empress® Direct was brought to market, it has not only become my standard filling material for the anterior region, but also a desirable alternative for posterior restorations. Moreover, the material has also proved to be excellently suitable for complete crown reconstructions in daily practice as a result of its harmonious shade match, wear resistance and excellent polishing properties.

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### Case presentation

A bitewing radiograph at the annual recall appointment revealed secondary caries at the distal margin of the crown of tooth 26. After having discussed the various restorative options suitable in his case, the 52-year-old patient decided to have a direct composite restoration.

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after | nachher

Starting situation: Full gold crown restoration with secondary caries at the distal margin

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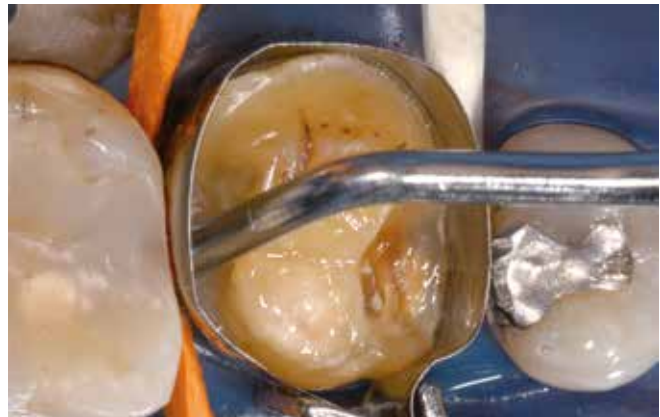
Situation after tooth preparation, placement of a wraparound matrix and adhesive application

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The contact points are established with Tetric EvoFlow® flowable composite.

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The crown is built up in increments using  
IPS Empress Direct Enamel A3.5.

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After the circular walls have been completed,  
the matrix is removed.

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The cusps are shaped immediately, making sure to  
correctly reproduce the occlusal morphology. This  
significantly reduces the effort and time required  
for finishing.

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Situation after completion of the composite build-up and coarse finishing

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Establishing of the occlusal contacts

Okklusale Kontakte etablieren

Postoperative situation

Postoperative Situation

Two years after completion of the treatment:  
The situation is unchanged.

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Lateral view two years after completion of  
the treatment

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# Prof. Dr Antonio Cerutti

“

Simplicity and high technology in every composite application.

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”

Professor and Director of Restorative Dentistry Department  
School of Dentistry | University of Brescia | P.le Spedali Civili 1 | 25123 Brescia | Italy  
cerant@tin.it | antonio.cerutti@unibs.it | www.restorativeonline.it

### Case presentation

Anterior restorations are among the most frequently performed treatments in daily practice. Large-scale defects represent a particular challenge in this context.

Adhesive restorative procedures with composite resin assist in creating an appropriate surface texture and reliably restoring dental esthetics.

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after | nachher

Starting situation: Tooth 11

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A rubber dam is placed to create a dry field.

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The cavity is prepared.

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Enamel etching

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Dentin composite is adapted to the tooth structure.

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Enamel material is placed along the incisal edge.

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OpraSculpt Pad is used to shape and contour the composite resin.

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Restoration prior to polishing

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Restoration after polishing

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Completed restoration after the rubber dam has been removed

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Final appearance after the application of surface structures during the second appointment

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Tooth 11: IPS Empress Direct  
Tooth 21: IPS e.max® crown


Zahn 11: IPS Empress Direct  
Zahn 21: IPS e.max® Krone

IPS Empress<sup>®</sup> Direct. The only  
Direct Composite that can be called Empress!

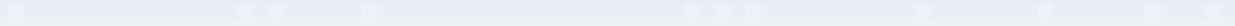The esthetics of a ceramic with the convenience  
of composite.

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**Ivoclar Vivadent AG**