# case

# book

Prof. Dr Jürgen Manhart  
Germany

Dr Markus Lenhard  
Switzerland

Dr Eduardo Mahn  
Chile

Dr Arnd Peschke  
Liechtenstein

Tetric EvoCeram®

**YEARS**  
OF CLINICAL  
EVIDENCE

Tetric EvoFlow®  
Tetric EvoCeram®  
Tetric EvoCeram® Bulk Fill

ivoclar  
vivadent®  
passion vision innovation

# 10 years of clinical evidence

A classic celebrates its birthday: Tetric EvoCeram®, the universal composite restorative, is marking a decade of clinical use. To date, more than 100 million restorations have been placed with the product. These restorations continue to look esthetic even after a number of years – a fact that has been confirmed by long-term studies.

Tetric EvoCeram Bulk Fill is the latest further development. It additionally contains the innovative, patented light initiator Ivocerin.

This Casebook presents a number of cases from internationally renowned dental professionals who value the outstanding properties of Tetric EvoCeram and Tetric EvoCeram Bulk Fill.

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# Prof. Dr Jürgen Manhart

“

I have used Tetric EvoCeram® for a decade. My patients' fillings look great, even years after placement.

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Defective composite restoration in tooth 12

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Final picture taken in 2005

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At the recall examination in 2014: The retention of shape and the esthetic appearance of the restoration are still excellent.

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# Dr Markus Lenhard

A handwritten signature in black ink, appearing to read 'M. Lenhard'.

“

I believe that Tetric EvoCeram® Bulk Fill is a very good composite material with excellent handling properties. Furthermore, it simplifies and expedites the layering procedure. You can almost say that, once you have worked with Tetric EvoCeram Bulk Fill, you no longer want to work with conventional composites.

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Starting situation: Defective amalgam filling in tooth 36, distal fracture and multiple cracks on the buccal cusps

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Postoperative situation

Postoperative Situation

Recall after one year: The clinical situation is unchanged.

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# Dr Eduardo Mahn

“

I have used Tetric EvoCeram® ever since it was launched. It is very reassuring to see that the quality of the restorations continues to be outstanding, even after a number of years. My experiences have been excellent not only with Tetric EvoCeram, but also with its predecessor, Tetric Ceram. Therefore, I expect similar performance using the further developed Tetric EvoCeram Bulk Fill.

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Pre operative situation

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Recall examination one week after placement,  
2007

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After seven years, the restoration features  
outstanding esthetics.

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## Case 2 | Fall 2

Defective composite restorations in teeth 46 (Z250/3M, six years old) and 47 (Heliomolar®/ Ivoclar Vivadent, 13 years old), showing a recent fracture

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Final picture taken in 2009

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Check-up in 2014: The dimensional stability of the restorations is still excellent.

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A comparison of the Z250 and Tetric EvoCeram restorations placed in the same tooth revealed clear differences with regard to wear and dimensional stability within a similar wear period.

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Examination of the Z250 restoration placed in 2003 after six years; wear facets are clearly visible (same situation as in Picture 1 on Page 10).

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The Tetric EvoCeram filling placed in 2009 at the recall examination after five years: Despite the fact that the restoration which the young patient received was an extensive first molar restoration, wear was minimal (same situation as in Picture 3 on Page 10).

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Starting situation, 2012

Ausgangssituation, 2012

Tetric EvoCeram Bulk Fill filling at the recall after one week: The surface polish was not ideal.

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At the check-up in 2014: Even though the polishing result had not been ideal upon treatment completion, the composite filling exhibited good surface quality at the recall after two years. No traces of discolouration were visible along the margins of the filling which had been placed in one 4-mm increment.

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## Case 4 | Fall 4

Although both Tetric Ceram fillings have been in the mouth of the patient for 10 (tooth 26) and 14 (tooth 27) years, no signs of marginal discolouration and only minor defects along the margins are visible.

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## Case 5 | Fall 5

Examination of the Tetric Ceram restorations twelve years postoperatively, in 2014. The restorations had been placed in 2002.

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Arnd Peschke

“

The systematic review of 10-year-old posterior restorations made of Tetric EvoCeram® yielded truly impressive results. The restorations showed close-to-perfect marginal quality. Virtually no material-related flaws were detected. Therefore, the vast majority of the restorations received a “Good” to “Excellent” rating. Only few deviations were found, which are probably due to my own shortcomings as a user. This material doubtless enables clinicians to create long-lasting, esthetic tooth restorations.

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## Case 1 | Fall 1

Starting situation: leaking composite restoration in tooth 36 (secondary decay located cervically)

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Immediately after restoration of the lesion: The fill-ing blends in harmoniously with the remaining dentition.

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Ten years later, the filling is still in an impeccable state.

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Preoperative situation involving a defective amalgam restoration in tooth 35

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Final photograph taken in 2004

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Check-up in 2014: Apart from slight discolouration in the fissure area, the filling still shows seamless integration with the adjacent tooth structure.

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## Case 3 | Fall 3

Starting situation: proximal caries lesion on the mesial aspect of tooth 37

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Photograph upon treatment completion in 2004

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At the recall in 2014: Except for a minor loss of lustre, the filling shows no shortcomings.

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Tooth 14 - preoperative situation

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Concluding picture dating from 2004

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Recall exam in 2014: Despite the patient's casual attitude towards oral hygiene, hardly any discolouration and only minor loss of some of the lustre were observed.

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Tooth 25 – preoperative situation

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Final photograph dating from 2004

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At the recall appointment in 2013: The filling shows minor marginal discoloration and a silky mat gloss, but no other shortcomings whatsoever.

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**Ivoclar Vivadent AG**