Programat[®] P710

Short Instructions | Kurzanleitung | Mode d'emploi : l'essentiel
Istruzioni d'uso – versione ridotta | Guía rápida | Guia Rápido

CE

Programat® P710

Short Instructions

Dear Customer

Thank you for having purchased the Programat® P710. In order to render the setting-up and the initial use of the furnace as easy as possible, we have prepared these short instructions.

These short instructions replace by no means the detailed Operating Instructions. We therefore kindly ask you to thoroughly read the Operating Instructions and follow the notes contained therein.

Enjoy working with the Programat P710.

1. Installation

Remove the furnace components from their packaging and place the unit on a suitable table. Please observe the instructions on the outer packaging. Make sure the voltage indicated on the rating plate complies with the local power supply. If this is not the case, the furnace must not be connected.

A Mounting the cooling tray

Remove both screws for the cooling tray (1) and the transport protection of the infrared camera.

B Place the cooling tray (2) on the frame plate (3).

Make sure that the cooling tray is correctly positioned on the frame plate.

C Secure the cooling tray with the two screws (1).


D Placing the firing plate

The firing plate (4) can now be placed on the firing plate holder (5).

E Mounting the furnace head

The furnace head is best mounted with the rear panel of the furnace pointing towards the user. Lift the furnace head with both hands as shown in the picture and place it on the furnace head mounting.

- F**  Mount the furnace head as shown in the picture until it audibly snaps into place. Make sure that the firing plate or the insulation is not damaged by mounting the furnace head.
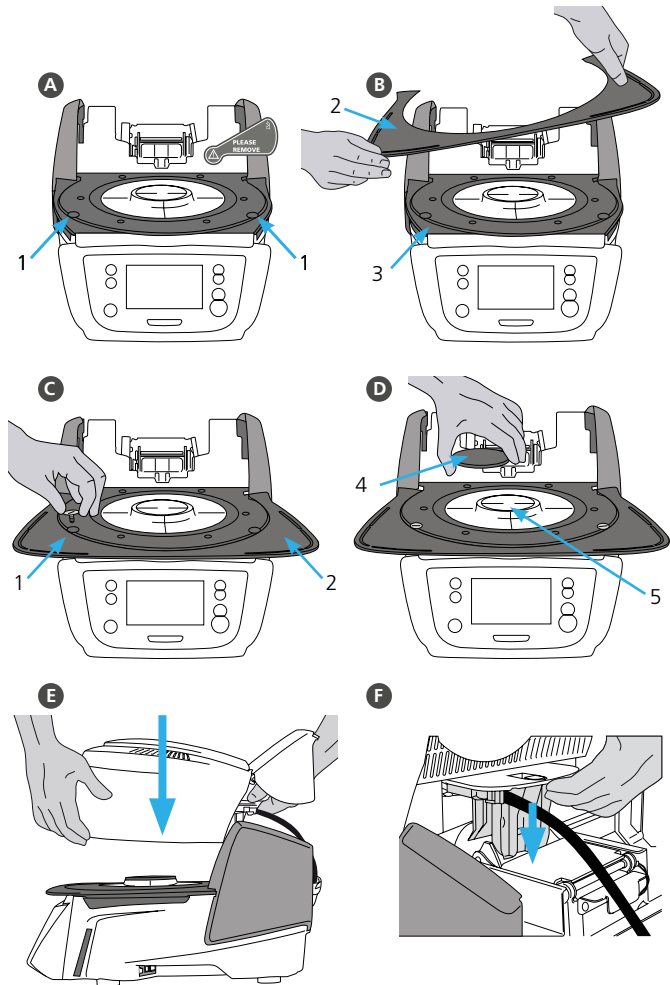


G Establishing the connections

Connect the cables of the furnace head with the furnace base.

Proceed as follows:



- Connect the vacuum hose (6).
- Insert the thermocouple plug (7). (Make sure that the polarity of the plug is correct.)
- Insert the heater plug (8).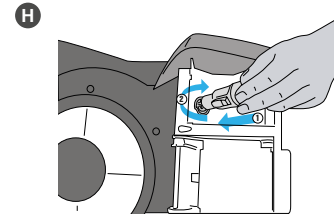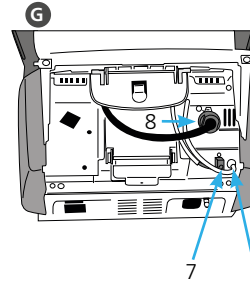

H Insert the heater plug into the intended connector. Secure the heater plug by rotating it 45°.

I Closing the covering hood

Once all cables are correctly connected to the furnace base, close the cover (9) to protect the connections until you feel it click into place.

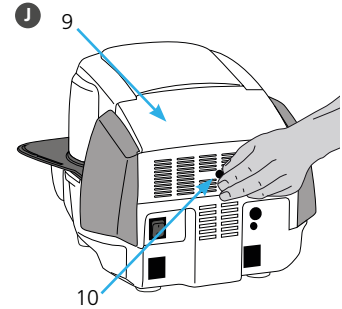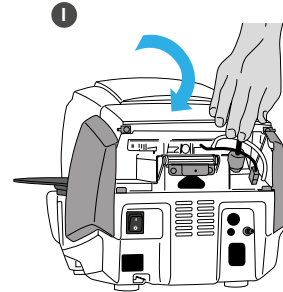J Mounting the connection cover

The connection cover (10) can now be mounted and secured with the screws.

The furnace may only be used with the covering hood closed and the connection cover mounted!


K Establishing additional connections

Power connection

Make sure that the voltage indicated on the rating plate complies with the local power supply. Subsequently, connect the power cord (11) with the power socket of the Programat (12).


Connecting the vacuum pump

Connect the vacuum pump plug (13) with the vacuum pump socket (14). We recommend using only a vacuum pump from Ivoclar Vivadent, since these pumps are especially coordinated with the furnace. If other pumps are used, please observe and do not exceed the indicated maximum power consumption.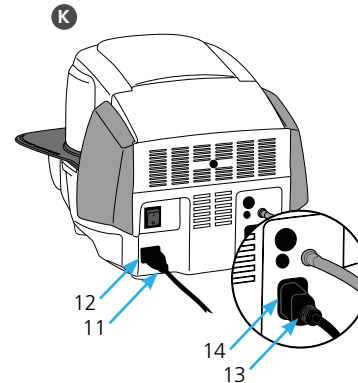

2. User interface

The Programat P710 has a wide-screen colour display. The keypad and the touch screen allow user-friendly handling. By tapping slightly with your fingertips on the touch screen, the buttons are activated and the furnace performs the requested function.

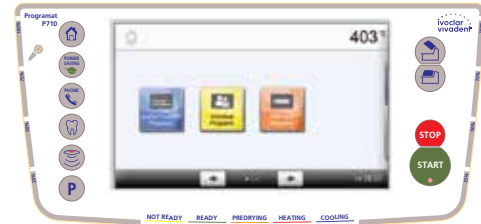Keypad:

Home

Returning to Home screen (main menu)

Power-saving key

This key activates the power-saving function.

Open furnace head

Close furnace head

START (Start LED)

Starts the selected program. The green LED indicates that a program is running. If the program is paused (1 x STOP), the Start LED flashes until renewed pressing of START results in the program being resumed.

STOP

A running program can be paused by pressing the STOP key and terminated by pressing the key twice. Movement of the furnace head can be stopped at any time by pressing STOP. Acoustic signals can be turned off by pressing the STOP key.

Program key

Pressing once: The currently selected program is displayed.

Pressing twice:

The program selection via number entry is accessed.

IRT key

With this key, the infrared image of the object can be displayed during the closing process.

Digital Shade Assistant key

With this key, the Digital Shade Assistant (DSA) can be started and closed.

Phone key


- If the furnace is not yet connected to a mobile phone, pressing this key results in the Bluetooth settings being displayed.
- If the furnace is already connected to a mobile phone, pressing this key results in the mobile phone menu being displayed.
- During an active phone call, this key can be used to fade the mobile phone screen in or out.

3. Operation

A Turning the furnace on

Connect the power cord with the wall socket.
Put the On/Off switch to position "I".

After the self-test, the display shows the Home screen. All functions of the Programat can be selected from this screen. Click the HOME key in order to return to the Home screen.

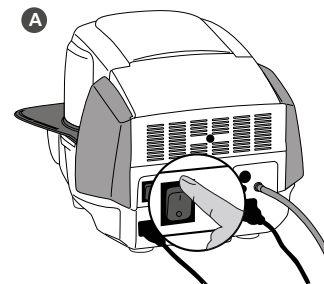B Selecting the program type

The furnace comprises multiple program types
a. Programs for Ivoclar Vivadent materials
b. 500 individually adjustable programs

C Selecting the program group

The program types are divided into program groups.
Each program group comprises 20 programs.

D Selecting the program

Select the required program.

Loading the furnace

Open the furnace head by pressing the OPEN FURNACE HEAD key and place the object to be fired with the firing tray in the furnace.

Programat Infrared Technology: Please read Chapter 6.3 of the Operating Instructions carefully before using the furnace for the first time. The notes regarding the Programat Infrared Technology must be observed!

E Starting the program

Press the START key to start the program. The LED lights up green. You can see the progress in the firing curve. At the end of the program, the furnace head opens automatically. As soon as the OSD lights up green, the furnace is available for the next program start.

4. Editing the firing parameters

Parameters are set or edited in two steps.

Example: Setting the holding temperature

- A** Press the [T] button.
- B** Enter the desired holding temperature and confirm by pressing the green button.

The holding temperature has been changed successfully. All parameters shown in the firing curve can be changed / edited the same way.


Changing program options

By pressing the [Options] button, the menu for additional program options opens.

Example: Open the write-protection program

- A** Press the [Options] button.
- B** Edit the desired option and return to the program view by pressing the [Close] button.

By pressing the [Arrow] buttons, you can change between the individual option pages.

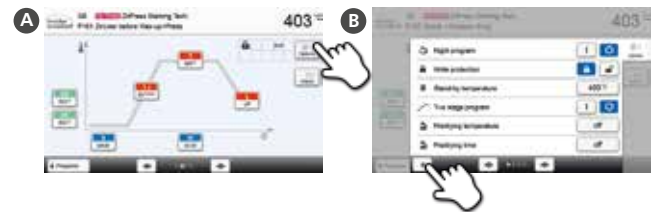Please refer to the **Operating Instructions** for more detailed information.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.