Ceramic Repair System Kit

EN Instructions for Use

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Instructions for Use

Description

The Ceramic Repair System Kit is a set of various components for the intraoral repair of defective ceramic- and composite-veneered fixed restorations. All-ceramic restorations can also be repaired with the Ceramic Repair System Kit.

Composition

The Ceramic Repair System Kit consists of the following components:

Tetric EvoCeram® – a light-curing nano-hybrid composite

The monomer matrix is composed of dimethacrylates (17-18 wt%). The fillers contain barium glass, ytterbium trifluoride, mixed oxide and prepolymer (82–83 wt%). Additional contents are additives, initiators, stabilizers and pigments (<1.0 wt%). The total content of inorganic fillers for the Bleach shades is 79–80 wt% or 60–61 vol%. For all the other shades, the content is 75–76 wt% or 53–55 vol%. The particle size of the inorganic fillers is between 40 nm and 3,000 nm with a mean particle size of 550 nm.

IPS Empress Direct® Opaque – a light-curing opaquer suitable for masking exposed metal surfaces

The monomer matrix is composed of dimethacrylates (53.9 wt%). The inorganic fillers include barium glass, ytterbium trifluoride, Ba-Al-fluorosilicate glass and spheroid mixed oxide (43.4 wt%). Initiators, stabilizers and pigments are additional contents (2.7 wt%). The total content of inorganic fillers is 23.2 vol% and the particle size ranges from 0.04 to 3.0 µm. The mean particle size is 0.7 µm.

Monobond® Plus – a primer designed for establishing an adhesive bond between luting composites and all indirect restorative materials

Alcohol solution of silane methacrylate, phosphoric acid methacrylate and sulphide methacrylate.

Heliobond – a light-curing bonding agent

Heliobond consists of Bis-GMA and tri-ethylene glycol dimethacrylate (99 wt.%), initiators and stabilizers (<1%).

Indication

Intraoral repair of defective ceramic- and composite-veneered fixed restorations

Contraindications

Use is contraindicated

- if a dry working field cannot be established or the prescribed application technique cannot be applied;
- if a patient is known to be allergic to any of the ingredients of Tetric EvoCeram, IPS Empress Direct Opaque, Monobond Plus and Heliobond.

Side effects

In rare cases, components of Tetric EvoCeram, IPS Empress Direct Opaque, Heliobond and Monobond Plus may lead to sensitization. The product should not be used in such cases.

Interactions

Tetric EvoCeram / IPS Empress Direct Opaque / Heliobond

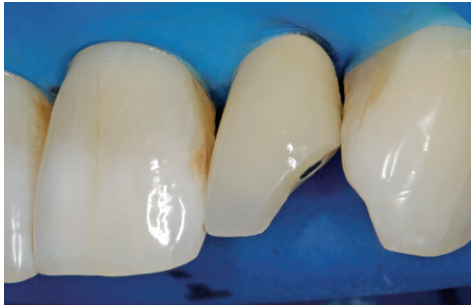
Phenolic substances such as eugenol / clove oil and wintergreen oil inhibit the polymerization of materials based on methacrylate. Materials containing such substances must not be used in conjunction with Tetric EvoCeram, IPS Empress Direct Opaque and Heliobond.

Discolouration may occur in combination with cationic mouthwashes, plaque disclosing agents and chlorhexidine.

Monobond Plus

Contamination of the fracture surfaces with saliva, blood or water must be absolutely avoided during the entire procedure.

Usage

1. Isolation

Appropriate isolation is required, preferably with a rubber dam (e.g. OptraDam® Plus).

2. Preparing the metal surface

In case of an exposed metal surface, roughen the surface with a diamond or corundum grinding instrument using water irrigation.

Alternatively, roughen the metal surface with an intra-oral sandblasting device (observe the instructions of the manufacturer).

3. Preparing the defective ceramic or composite surface

Bevel the margins of the defective ceramic or composite surface (1-2 mm) with a fine-grained diamond grinding instrument using water irrigation.

Subsequently, rinse thoroughly with water and dry with oil-free air.

Note:

If tooth structure is affected by this repair, these areas must be pretreated with a dentin/enamel adhesive.

4. Placing the matrix

If the restoration includes proximal areas, place a matrix band and secure it with a wedge.

5. Monobond Plus

Use a brush to apply Monobond Plus to the exposed ceramic, metal and composite surfaces and allow to react for 60 seconds.

Subsequently, dry with oil-free air. (Do not rinse with water.)

6. Applying Heliobond

Apply a thin layer of Heliobond to the entire surface area which needs to be repaired. Remove any excess with compressed air.

Then, light-cure for 10 seconds
(Bluephase® Style, Bluephase 20i “High Power Program”).

7. Applying the opaquer

Apply IPS Empress Direct Opaque to the prepared metal surface only. The layer applied should have a thickness of maximum 0.5 mm.

Then, light-cure for 20 seconds
(Bluephase Style, Bluephase 20i “High Power Program”).
If the surface is still not sufficiently masked, apply a second layer of IPS Empress Direct Opaque and light-cure again.

8. Restoring the restoration with Tetric EvoCeram

Apply Tetric EvoCeram in increments of max. 2 mm and adapt the material with a suitable instrument (e.g. OptraSculpt Pad).

Light-cure each layer individually for 10 seconds (Bluephase Style, Bluephase 20i "High Power Program").

9. Finishing and polishing

Remove excess with a suitable fine-grained diamond grinding instrument.

Polish restorations to a high gloss using silicone polishers (e.g. Astropol® or OptraPol®).

10. Restoration after repair

Warning

Prevent any contact of uncured Tetric EvoCeram, IPS Empress Direct Opaque and Monobond Plus with skin, mucous membrane and eyes.

In an unpolymerized state, these products may cause slight irritation and lead to a sensitization against methacrylates.

Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Monobond Plus:

- Flammable
- Irritating
- The product contains ethanol!
Keep away from ignition sources.

Shelf life and storage

- Do not use the individual components of Ceramic Repair after the indicated expiry date.
- Observe the storage temperature of all components: 2–28 °C (36–82 °F)
- Shelf life: see imprint on syringe/bottle/cartridge
- Tetric EvoCeram, IPS Empress Direct Opaque, Heliobond: Close the products immediately after use. Exposure to light causes premature polymerization.
- Do not disinfect syringes with oxidizing disinfectants.
- Monobond Plus: close immediately after usage.

Keep out of the reach of children!

For use in dentistry only.

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.