

Cervitec® F




Professional protection in the dental practice with Cervitec F

In the dental practice, the protective varnish Cervitec F can be applied in a targeted fashion to highly susceptible areas. The varnish contains fluoride and chlorhexidine and provides a professional basic protection. In this way, Cervitec F supports your oral hygiene measures at home.

Your dentist and his team will examine your personal situation and advise you.



Cervitec® Gel

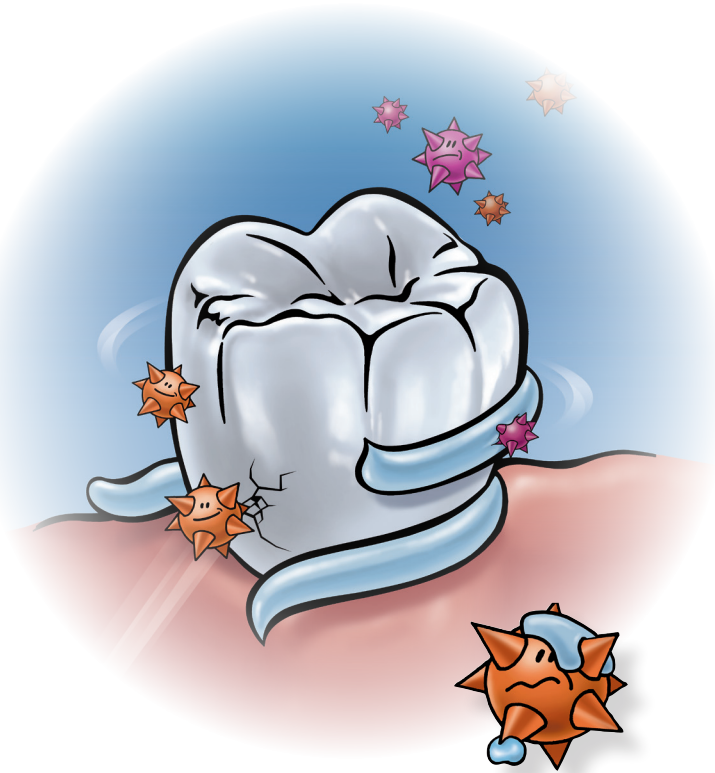

Cervitec® Gel



Beautiful teeth - for a smile at the world

All teeth, whether natural or filled, and tooth replacements require regular inspection and dental care measures throughout life.

Your benefits

Cervitec Gel and Cervitec F support you in keeping your teeth, implants or other restorations in the mouth healthy for a long time. If you have any questions, please do not hesitate to ask your dental team.

Package sizes Cervitec Gel

Single Tube Cervitec Gel, 20g
Single Tube Cervitec Gel, 50g

Cervitec® Gel



Healthy gums,
healthy teeth,
healthy mouth

*A smile connects –
Beautiful, healthy teeth and
healthy gums for a lifetime*

In cooperation with the Center for
Dentistry and Oral Medicine, University
Clinic Jena, Friedrich-Schiller-University
of Jena, Germany

**ivoclar
vivadent**
passion vision innovation

**ivoclar
vivadent**
passion vision innovation

640085/2018-01-19/EN


Cervitec® Gel

Healthy gums, healthy teeth, healthy mouth

There are a lot of different bacteria in the mouth. They form plaque on the gums, teeth and tooth replacements. Certain types of these bacteria are harmful to the gums. Others attack the teeth. Usually, you keep the harmful bacteria in check with your daily oral hygiene measures.

Personal challenges

Crowded teeth, fixed braces or tooth replacements make it difficult to clean certain areas: harmful bacteria settle and multiply in these areas. They may cause inflammation of the gums, damage to the teeth (tooth decay) and odours. Other factors may also play a role in the case of inflammation and gum bleeding. Your dentist and his team will advise you on the possible causes and initiate the suitable therapy.

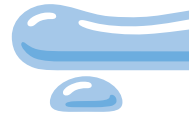


Cervitec® Gel

Keep bacteria at a healthy rate

Problem areas in the mouth require special protection, for example with Cervitec Gel. Chlorhexidine, which has been a tried-and-tested substance for years, reduces harmful bacteria. Consequently, less plaque is formed. Inflammation of the gums and mucous membranes ebbs away. The tissue can recover. Bacterial growth on tooth replacements is reduced. Your breath will improve.

Additional protection with fluoride

Like your toothpaste, Cervitec Gel also contains fluoride, which has been clinically proven for decades. It strengthens your teeth and protects them from tooth decay. Therefore, you can use the gel to brush your teeth in the evening.

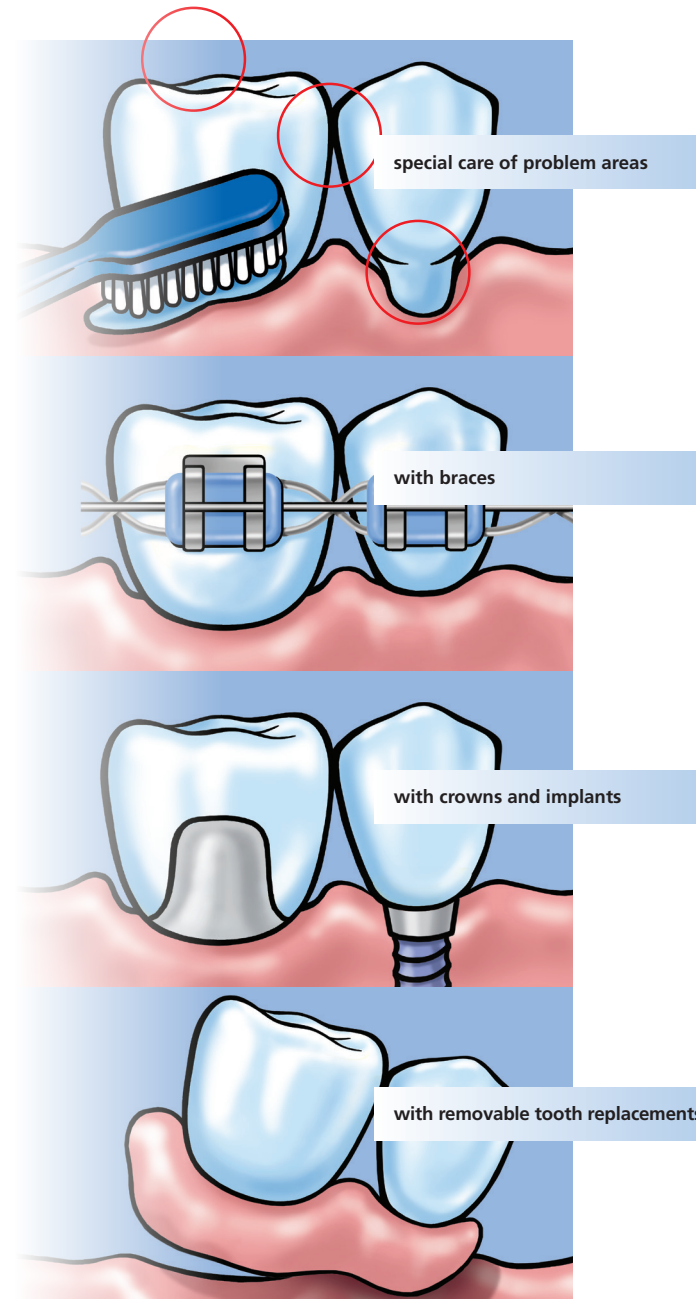

Cervitec® Gel

Use an oral health care gel

- Depending on your needs, you may apply Cervitec Gel
- ▶ directly on the gums, oral mucous membranes or the inner aspects of removable tooth replacements
 - ▶ on the interdental brush. Clean the space between teeth and fixed tooth replacements – for example crowns, bridges or implants – with the gel
 - ▶ brush your teeth with Cervitec Gel as usual

Tip You can support the care effect. Do not rinse: simply spit it out – done!

Application time

- ▶ Until the symptoms subside, unless prescribed otherwise.
- ▶ Even if the gel is used for more than 4 weeks, no staining of teeth or impaired sense of taste is to be expected.

