# Programat® EP 5010

Short Instructions | Kurzanleitung |  
Mode d'emploi : l'essentiel |  
Istruzioni d'uso – versione ridotta |  
Guía rápida | Guia Rápido

CE

# Programat® EP 5010

## Short Instructions

### Dear Customer

Thank you for having purchased the Programat® EP 5010. In order to render the setting-up and the initial use of the furnace as easy as possible, we have prepared these short instructions.

**These short instructions replace by no means the detailed Operating Instructions. We therefore kindly ask you to thoroughly read the Operating Instructions and follow the notes contained therein.**

Enjoy working with the Programat EP 5010.

# 1. Installation

Remove the furnace components from their packaging and place the unit on a suitable table. Please observe the instructions on the outer packaging. Make sure the voltage indicated on the rating plate complies with the local power supply. If this is not the case, the furnace must not be connected.

## A Mounting the cooling tray

Remove both screws (1) for the cooling tray and the transport protection of the infrared camera.

B Place the cooling tray (2) on the frame plate (3). Make sure that the cooling tray is correctly positioned on the frame plate.

C Secure the cooling tray with the two screws (1).

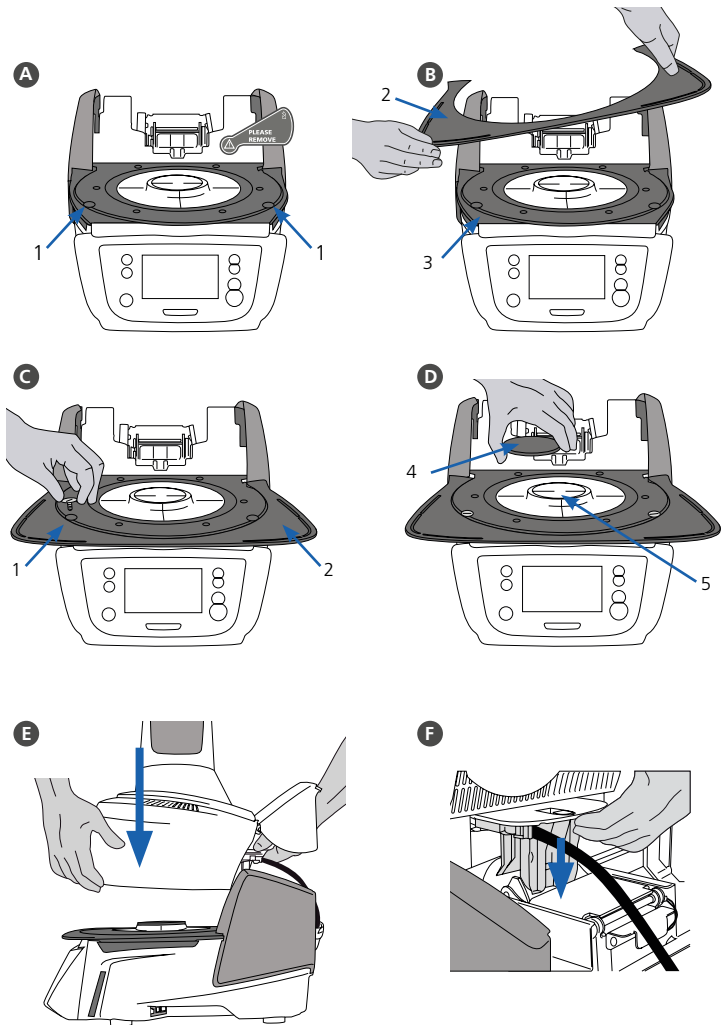
## D Placing the firing plate

The firing plate (4) can now be placed on the firing plate holder (5).

## E Mounting the furnace head

The furnace head is best mounted with the rear panel of the furnace pointing towards the user. Lift the furnace head with both hands as shown in the picture and place it on the furnace head mounting.

F  Mount the furnace head as shown in the picture until it audibly snaps into place. Make sure that the firing plate or the insulation is not damaged by mounting the furnace head.



### G Establishing the connections

Connect the cables of the furnace head with the furnace base.

Proceed as follows:

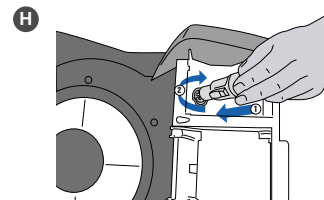
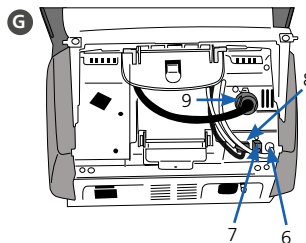
- Connect the vacuum hose (6).
- Insert the thermocouple plug (7). (Make sure that the polarity of the plug is correct.)
- Insert the press drive plug (8).
- Insert the heater plug (9).

H Insert the heater plug into the intended connector. Secure the heater plug by rotating it 45°.

### I Closing the covering hood

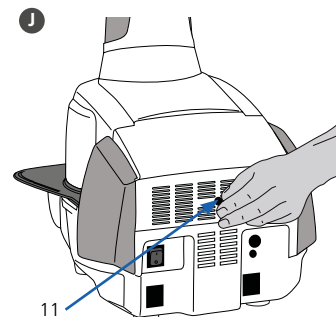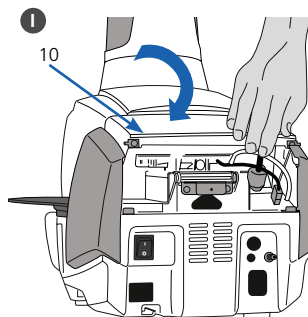Once all cables are correctly connected to the furnace base, close the cover (10) to protect the connections until you feel it click into place.

### J Mounting the connection cover

The connection cover (11) can now be mounted and secured with the screws.

The furnace may only be used with the covering hood closed and the connection cover mounted!

### K Establishing additional connections

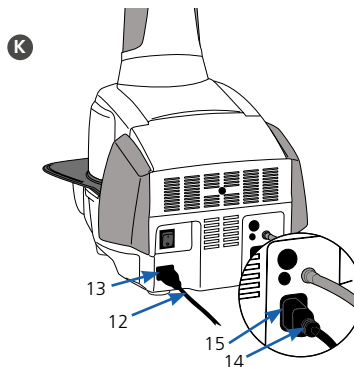
#### Power connection

Make sure that the voltage indicated on the rating plate complies with the local power supply. Subsequently, connect the power cord (12) with the power socket of the Programat (13).

#### Connecting the vacuum pump

Connect the vacuum pump plug (14) with the vacuum pump socket (15).

We recommend using only a vacuum pump from Ivoclar Vivadent, since these pumps are especially coordinated with the furnace. If other pumps are used, please observe and do not exceed the indicated maximum power consumption.

## 2. User interface

The Programat EP 5010 has a wide-screen colour display. The keypad and the touch screen allow user-friendly handling. By tapping slightly with your fingertips on the touch screen, the buttons are activated and the furnace performs the requested function.

### Keypad:

#### HOME key

Switch to Home screen (main menu).

#### PROGRAM key

**Pressing once:** The currently selected program is displayed.

**Pressing twice:** The program selection via number entry is accessed.

#### CHANGE OPERATING MODE key

By pressing this key, you can switch between the firing and the press mode.

#### IRT key

With this key, the infrared image of the object can be displayed during the closing process.

#### POWER SAVING key

This key activates the power-saving function.

#### OPEN FURNACE HEAD key

#### CLOSE FURNACE HEAD key

#### START key (Start LED)

Starts the selected program. The green LED indicates that a program is running. If the program is paused (1 x STOP), the Start LED flashes until renewed pressing of START results in the program being resumed.

#### STOP key

A running program can be paused by pressing the STOP key and terminated by pressing the key twice. Movement of the furnace head can be stopped at any time by pressing STOP. Acoustic signals can be turned off by pressing the STOP key.

# 3. Operation

## Start-up

Connect the power cord with the wall socket and put the On/Off switch at the rear of the furnace to position "I".

After the self-test is completed, the furnace will automatically display the screen to select the desired operating mode. Select the firing or press mode.

### 3.1 Operation in the FIRING mode

#### Step 1:

##### Select the program type.

The furnace offers several types of programs:

- firing programs for Ivoclar Vivadent materials
- 500 free, individually adjustable firing programs

#### Step 2:

##### Select the program group.

The program types are subdivided into program groups. Each program group consists of 20 programs.

#### Step 3:

##### Select the desired firing program.

In this screen, firing programs can be started or edited.

#### Step 4:

##### Edit the firing program.

To edit a parameter, press the respective button and enter the desired value.

Subsequently, confirm the entry by pressing the green button.

When the furnace is delivered ex factory, Ivoclar Vivadent programs are write-protected.

The write-protection can be deactivated in the Options menu, where additional program options are available.

#### Step 5:

##### Load the furnace and start the program.

Open the furnace head using the OPEN FURNACE HEAD key and position the firing tray with the object in the furnace. Activate the program by pressing the START key.

Programat Infrared Technology: Please read the information provided in Chapter 6.5 of the Operating Instructions thoroughly before the initial start-up of this furnace. The notes regarding the Programat Infrared Technology must always be observed!

### 3.2 Operation in the PRESS mode

#### Step 1:

##### Select the program type.

The furnace offers several types of programs:

- press programs for Ivoclar Vivadent materials
- 20 free, individually adjustable press programs

#### Step 2:

##### Select the desired press program.

#### Step 3:

##### Load the furnace and start the program.

If the IRT investment ring analysis is active, please follow the instructions on the display and load the furnace.

If the IRT investment ring analysis is inactive, select the desired investment ring size.

### Editing press programs

The Ivoclar Vivadent press programs are equipped ex factory with the optimum parameters for Ivoclar Vivadent pressed ceramic materials. These programs are write-protected and cannot be edited. Individual press programs can be edited in the same way as the firing programs (see 3.1).

Please see the Operating Instructions for more detailed information.

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## **Ivoclar Vivadent AB**
## **Ivoclar Vivadent Limited**
## **Ivoclar Vivadent, Inc.**