Instructions for Use EN

PRODUCT DESCRIPTION
Pd-based dental metal-ceramic alloy, Type 4

INDICATIONS*
Crowns, Telescope Crowns, Conus Crowns, Bridges, Wide Bridges, Cast Posts-/Cores, Bars, Attachments, Implant Retained Superstructures, Partial Dentures

WAXING/MODELLATION
Design the framework in a reduced anatomic shape taking the planned veneer into consideration. Single crown thickness of 0.3 mm. Abutment crowns require a minimum thickness of 0.5 mm. Ensure the framework design provides adequate support for the veneering material. Avoid sharp angles. Connectors must have the required dimensions to provide resistance to deformation. Create large surface areas for planned soldering, with a gap of 0.05–0.2 mm.

SPRUNG
Pre-fit the modeled single-tooth restoration or bridge framework with sprues of a suitable size. In general the reservoir, sprue leads, and connector sprues, whether per shaped or traditional, must be sized according to the specific technique used. When using the direct or indirect technique be sure that the reservoir is positioned in the heat center. The reservoir and the casting should be a maximum of 2.5–3.0 mm in length and width. The wax pattern including the sprues must be weighed in grams in order to determine the needed amount of alloy. Wax conversion formula: wax weight (gram) x alloy density = grams of alloy required.

INVESTING
Use phosphate-bonded investment material. Follow the manufacturer’s instructions.

PREHEATING/ BURN-OUT
Recommended burn-out temperature: 900 °C

MELTING AND CASTING
Torch: Propane 0.35 bar; Oxygen 0.7 bar
Other specifics may be required by the type of casting machine. It is recommended to use a separate and clean ceramic crucible for each alloy. Preheat the ceramic crucible in the burnout furnace. The recommended ratio of used material to new material is 1:1. Do not use flux.
Casting Temperature: 1400 °C

FRAMEWORK FINISHING
After final cooling, carefully clean the casting with aluminum oxide (Al₂O₃). Do not use a hammer for divesting. Finish the casting with carbide burs and/or with ceramic-bonded grinding instruments. Blast the surface with 50–110 micron aluminum oxide (Al₂O₃) at 2.0 bar. Subsequently, steam clean or ultrasonic clean with distilled water or ethanol and dry the framework.

OXIDATION
The framework can be fired tray providing adequate support. To achieve a uniform result follow the oxidation cycle.
Temperature: 930 °C. Holding time: 5 min. Vacuum: No
If the oxide layer is stained, grind and blast the surface again. Repeat the oxide firing. Use the appropriate ceramic veneering material, following the manufacturer’s instructions.
Highest recommended firing temperature: 1000 °C

HEAT TREATMENT
Hardening: 600 °C for 15 min; bench cool

SOLDERING AND FLUX WELDING
The soldering gap should be no wider than the thickness of the soldering material. Allow the soldered casting to cool slowly. Use flux sparingly.
Pre Solder: Porta Solder 1090 W Flux: High Fusing Bondal Flux
Lot vor dem Brand / Flussmittel: Porta OP Solder W-2 Bondal Flux
Lot nach dem Brand: Porta OP Solder W-2 Flux: Bondal Flux
Laser-Schweiß-Draht: Porta P6 Flux: Laser-Schweiß-Draht: Porta P6

POLISHING
Carefully remove any oxide and flux residue. Smooth the metal surfaces with rubber polishers. Polish to a high gloss finish using polishing paste. Subsequently, clean using ultrasonic cleaning equipment or careful steam cleaning.

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