EN Instructions for Use

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Description

Helioseal® is a light-curing, white-shaded fissure sealant.

Composition

Helioseal consists of Bis-GMA and triethylene glycol dimethacrylate (> 97 wt%), as well as titanium dioxide (2 wt%). Additional contents are stabilizers and catalysts (< 1 wt%).

Indication

Helioseal is used to seal pits, fissures, and Foramina Caeca.

Contraindication

- If the patient is known to be allergic to any of the material's ingredients.
- If a dry working field cannot be established.

Side effects

In individual cases, contact allergies may occur. According to today's standard of knowledge, systemic side effects are not known.

Interactions

None known to date.

Rx ONLY

Date information

prepared

557037/2013-12-05/WE3/Rev1

CE 0123

Manufacturer
Ivoclar Vivadent AG
FL-9494 Schaan/Liechtenstein
www.ivoclarvivadent.com

ivoclar
vivadent®
clinical

Step-by-step application procedure

Shake the Helioseal bottle well before use. Open the bottle only immediately before use to prevent premature polymerization by light.

1. Thoroughly clean the enamel surface to be sealed.

2. Isolate the working field, preferably with a rubber dam.

3. Apply an etching gel, e.g. Email Preparator, and let it react for 30 to 60 seconds.

4. Rinse thoroughly.

5. Dry with water- and oil-free air. The etched enamel should have a mat white appearance. Avoid contamination of the etched surface with saliva.

6. Apply Helioseal directly with the disposable cannula or a disposable brush, and disperse.

7. Wait for approx. 15 sec. Then cure the sealant with a suitable polymerization light (e.g. Bluephase®) for 20 sec.

8. Check seal and occlusion.

Recommendation

We recommend subsequent fluoride application (e.g. Fluor Protector) to provide optimum protection after sealing.

Warning

Avoid contact of unpolymerized material with skin/mucous membrane or eyes. Unpolymerized Helioseal may cause slight irritations and, in rare cases, may lead to a sensitization against methacrylates. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage

- Close Helioseal bottle immediately after use.
- Store material at 2–28 °C (36–82 °F).
- Shelf life: see date of expiration.
- Do not use the material after the indicated date of expiration.

Keep out of the reach of children

For use in dentistry only.

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.