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### Brukerveiledning

### Produktinformatie

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## Instructions for Use

### Description

Helioseal F is a light-curing, white-shaded fissure sealant featuring fluoride release.

### Composition

The monomer matrix consists of Bis-GMA, urethane dimethacrylate, and triethylene glycol dimethacrylate (58.6 wt%). The fillers are highly dispersed silicon dioxide and fluorosilicate glass (40.5 wt%). Additional contents are titanium dioxide, stabilizers and catalysts (< 1 wt%).

### Indication

Helioseal F is used to seal pits, fissures, and foramina caeca.

### Contraindication

- If the patient is known to be allergic to any of the materials ingredients.
- If a dry working field cannot be established.

### Side effects

In individual cases, contact allergies may occur. According to today's standard of knowledge, systemic side effects are not known.

### Other interactions

None known to date

### Complies with / entspricht: ISO 6874, Type 2

ANSI/ADA Spec. No 39, Type 2

The ADA Council on Scientific Affairs' Acceptance of Helioseal is based on its finding that the product is effective for use in sealing pits and fissures in teeth.

For dental use only.

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

CE 0123

555439/0510/WE

ivoclar  
vivadent®  
clinical

## Step-by-step application procedure

1. Thoroughly clean the enamel surface to be sealed.

2. Isolate the working field, preferably with a rubber dam.

3. Apply an etching gel, e.g. Email Preparator, and let it react for 30 to 60 seconds.

4. Rinse thoroughly.

5. Dry with water- and oil-free air. The etched enamel should have a mat white appearance. Avoid contamination of the etched surface with saliva.

6. Apply Heliocisal F directly with the disposable cannula or a disposable brush, and disperse.

7. Wait for approx. 15 seconds. Then cure the sealant with a suitable polymerization light (e.g. bluephase®) for 20 seconds.

8. Check seal and occlusion.

## Warning

Avoid contact of unpolymerized material with skin/mucous membrane or eyes. Unpolymerized Heliocisal F may cause slight irritation and, in rare cases, may lead to a sensitization against methacrylates. Commercial medical gloves do not

provide protection against the sensitizing effect of methacrylates.

## Special note

If Heliocisal F is applied from the Cavifil directly in the mouth of the patient, we recommend using this Cavifil only for one patient due to hygienic reasons (prevention of cross-contamination between patients). The same applies to the application tips of the syringe.

## Storage

- Close Heliocisal F syringe immediately after use
- Store material at 2–28 °C (36–82 °F).
- Shelf life: see date of expiration.
- Do not use the material after the indicated date of expiration.
- Syringes or Cavifils should not be disinfected with oxidizing disinfection agents.

**Keep out of the reach of children!**  
**For use in dentistry only!**

## Date information prepared

02/2010, Rev. 2

## Manufacturer

Ivoclar Vivadent AG  
FL-9494 Schaan / Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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**Ivoclar Vivadent UK Limited**
**Ivoclar Vivadent, Inc.**