



IPS e.max® ZirCAD MT BL/LT 0

All ceramic,
all you need.

BASIC infiltration with IPS e.max® ZirCAD MT/LT Colouring Liquids A–D

- 25  IPS e.max® ZirCAD MT/LT Colouring Liquids (A–D)
- + 1  IPS e.max® ZirCAD Colouring Liquid Indicator red



1

1x with brush No. 5



2

1x with brush No. 5



3



1x with brush No. 5

4



1x with brush No. 5

5



1x with brush No. 5

6



Drying time
before the individual
advanced infiltration





ADVANCED infiltration with IPS e.max® ZirCAD MT/LT Colouring Liquids A–D

Recommendations



for shading the Effect shades
with IPS e.max® ZirCAD Colouring
Liquid Indicator:

25  IPS e.max® ZirCAD MT/LT
Colouring Liquid grey, violet
+
1  IPS e.max® ZirCAD Colouring
Liquid Indicator yellow



25  IPS e.max® ZirCAD MT/LT
Colouring Liquid orange
+
1  IPS e.max® ZirCAD Colouring
Liquid Indicator blue



25  IPS e.max® ZirCAD MT/LT
Colouring Liquid blue
+
1  IPS e.max® ZirCAD Colouring
Liquid Indicator blue + yellow (= green)



Apply grey in the incisal and marginal ridge area
for a lifelike in-depth effect

1x with brush No. 1 or 3



Translucent effects with blue in the incisal area

1x with brush No. 1 or 3



Translucent effect with violet in the incisal
and cervical area

1x with brush No. 1 or 3



10 Drying time
before sintering