

PrograMill PM7

PrograMill PM7 Softwareupdate V80.56 Changelog

New features and bug fixes:

- *new feature*: In case of emergency stop, Z-axis moves out of object and spindle stops
- *new feature*: Pre-check before importing new jobs
- *new feature*: Optimized communication to the suction unit
- *new feature*: Leave spindle clamp open during spindle probe
- *new feature*: Inform the user about "Auto mode" off
- *new feature*: Give feedback at the end of "Restoring the coolant system" process
- *fixed critical bug*: Calibration values correctly displayed
- *fixed critical bug*: Fixed wrong interval of the cleaning reminder
- *fixed critical bug*: Fixed error ESTOP_SOFTWARE_ENABLE shown accidentally
- *fixed bug*: "Eject" function in done job list
- *fixed bug*: Optimizations regarding error ERR_UI_WD_7103: "Storage utilization too high."
- Introduction of new upper warning and error thresholds for water flux