

IPS Ivocolor®

Combination table for staining the IPS veneering materials and the all-ceramic materials of the IPS e.max® system*

Kombinationstabelle für die Bemalung von IPS Verblendkeramiken und Vollkeramiken des IPS e.max® Systems*

		A–D tooth shade Zahnfarbe																				
		BL1	BL2	BL3	BL4	A1	A2	A3	A3.5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4	
IPS Ivocolor Shade	Shade 0	■	■	■	■																	
	Shade 1					■																
	Shade 2						■	■	■													
	Shade 3									■												
	Shade 4										■	■										
	Shade 5												■	■								
	Shade 6															■	■	■	■			■
	Shade 7																			■	■	
	Shade 8																					

* Other shade combinations apply when using IPS e.max ZirCAD LT discs in the LT group shades.

Bei der Verwendung von IPS e.max ZirCAD LT Discs in den LT-Gruppenfarben gelten andere Farbkombinationen.













see instructions

IPS Ivocolor®

Combination table for IPS Ivocolor® components

Kombinationstabelle für IPS Ivocolor® Komponenten

Components Komponenten	Initial mixing Erstmaliges Anmischen			Refreshing of dried-out materials Aufrischung von eingetrockneten Massen		
	IPS Ivocolor Mixing Liquid allround	IPS Ivocolor Mixing Liquid longlife	IPS Ivocolor Essence Fluid	IPS Ivocolor Mixing Liquid allround	IPS Ivocolor Mixing Liquid longlife	IPS Ivocolor Essence Fluid
 IPS Ivocolor Essence						
 IPS Ivocolor Shade	✓	✓	✗	✓	✓	✗
 IPS Ivocolor Glaze Powder/ FLUO	✓	✓	✗	✓	✓	✗
 IPS Ivocolor Glaze Paste/ FLUO	✓	✓	✗	✓	✓	✗

CE 0123

Rx only



Manufacturer:
Ivoclar Vivadent AG, 9494 Schaan/Liechtenstein
www.ivoclarvivadent.com



see instructions

Date information prepared: 2019-10, Rev. 1

668431/d-e

ivoclar
vivadent[®]
technical