# PrograPrint Cure

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# English

## 1 About this document

These Short Instructions are part of the overall documentation for the device. They are only valid in combination with the Operating Instructions of the device ("Documents of equal applicability", page 6).

### **WARNING! Improper use of the device.**

Risk to the patient and the user.

- ▶ The Short Instructions contain important information on the safe handling of the device: Operating Instructions and Short Instructions must be read thoroughly prior to installation and operation.

These Short Instructions will help you to quickly familiarize yourself with the use of PrograPrint Cure.

Should you lose these Short Instructions, you can download them from [www.ivoclarvivadent.com](http://www.ivoclarvivadent.com).

## 1.1 Signs and symbols

### 1.1.1 Warnings and additional information

#### Warnings

Warnings are used in these Operating Instructions to warn you of a risk of injury to persons and/or damage to property.

1. Please always read and observe these warnings.
2. Follow all measures marked with the warning symbol and warning word.

Depending on the severity and probability of the danger, the following warning levels are distinguished:

Warning symbol	Warning word	Danger level	Consequences resulting from non-observance
	DANGER	Immediate imminent danger	Death, severe personal injury
	WARNING	Potential danger	Death, severe personal injury
	CAUTION	Potential danger	Slight personal injury
-	NOTICE	Potential danger	Damage to property

Explanation of the structure of a warning:

 **WARNING WORD! Name of source of danger, cause of hazard or type of risk.**

Consequences of non-compliance with instructions.

- ▶ Action in order to avoid danger.

Example of a warning:

 **DANGER! Cleaning work during operation. Contact with voltage-carrying components.**

Danger of electric shock.

Damage to the device.

- ▶ Perform cleaning work only when the device is switched off and disconnected from the power supply.

**Additional information**

Symbol	Meaning
	Additional information, e.g. for better understanding, for simplifying work flows or for further information.

**1.2 Documents of equal applicability**

 For details on the documents of equal applicability, please contact your local trade partner or go to the download section of Ivoclar Vivadent at <http://www.ivoclarvivadent.com>.

Document	Explanation
Operating Instructions PrograPrint Cure	Comprehensive information on the use of PrograPrint Cure
Instructions for Use for the different materials	Information about the use of the materials to be processed
Operating Instructions on the use of 3D printer PrograPrint PR5	Information about requirements that must be observed for the transport, set-up and use of PrograPrint PR5
Operating Instructions for the cleaning unit PrograPrint Clean	Information about requirements that must be observed for the transport, set-up and use of PrograPrint Clean
Instructions for Use for CAMbridge	Manual for the CAM software "CAMbridge"
Short Instructions for: <ul style="list-style-type: none"> <li>• PrograPrint PR5</li> <li>• PrograPrint Clean</li> <li>• PrograPrint Cure (this document)</li> </ul>	Excerpt of operation-relevant topics from the respective Operating Instructions

---

## 2 Safety

### **WARNING! Improper use of the device.**

Hazard to the user.

- ▶ The safety chapter contains important information on the safety of the device: Please read it thoroughly prior to installation and operation.
- 

The device is built according to state-of-the-art technology and recognized safety regulations. Nevertheless, operation bears the risk of injury for the user or third parties. Moreover, impairments of the device and other property are possible.

---

### 2.1 Intended use

PrograPrint Cure has been designed for use in dental technology for curing and post-curing the following materials:

- Ivoclar Digital PrograPrint materials:
  - ProArt Print Wax (burn-out material for the press and casting technique)
  - ProArt Print Model (material for the fabrication of the dental working models)
  - ProArt Print Splint (material for the fabrication of dental drilling templates and splints)
- Dental technical, light-curing laboratory composites (e.g. SR Nexco)
- Dental technical, light-curing materials

Use PrograPrint Cure exclusively for this purpose.

In the case of damage caused by improper use or failure to observe the Operating Instructions, all liability and guarantee claims are void.

Please note that only the materials and cleaning agents approved by the manufacturer ensure processing without damage to the machine.

---

All information regarding material and indication extensions corresponds to the state of knowledge at the time of going to print. For additional details, go to

<http://www.ivoclarvivadent.com>.

The post-curing of 3D-printing materials from other manufacturers is not recommended due to the lack of coordination with this procedure.

---

### 2.1.1 Potential improper use

The device is not suitable for post-curing procedures outside of the dental field.

The following is considered as misuse of the system:

- Non-compliance with the specified intended use, intended operator specifications or intended environment
- Modifications, maintenance and repairs without prior authorization by Ivoclar Vivadent
- Operation with inappropriate parameters

Improper use of the devices can lead to the following:

- Danger for patients and operating personnel
- Impairment of the operability of the devices

Ivoclar Vivadent assumes no liability for damage resulting from improper use.

Typical misapplications of the device to be avoided:

Improper use	Consequences
Post-curing of non-approved materials	Insufficient material properties in terms of mechanics, colour, precision, biocompatibility
Post-curing of printed objects / objects in an articulator	<ul style="list-style-type: none"> <li>• Articulator does not fit into the chamber, door cannot be closed</li> <li>• Device cannot be used</li> <li>• Damage to the device</li> </ul>
Post-curing of non-dental objects	Insufficient material properties in terms of mechanics, colour, precision, biocompatibility due to different volume, weight, layer thickness, materials, etc. of the objects
Incorrect cleaning and disposal	<ul style="list-style-type: none"> <li>• Skin irritations</li> <li>• Environmental damage</li> <li>• Damage to the device</li> </ul>
Switching off the device by the user during operation or opening the door whilst a program is running	<ul style="list-style-type: none"> <li>• Interruption or cancellation of the curing process/program</li> <li>• Objects not completely cured</li> <li>• Insufficient material properties in terms of mechanics, colour, precision, biocompatibility</li> </ul> <p>Note: If a curing job in progress is cancelled, it cannot be continued. It is not possible to perform a post-curing procedure after a program cancellation as this would lead to an undesirable change in the material properties.</p>

### 2.1.2 Intended user

The device may only be operated by dental technicians and qualified dental staff for the fabrication of dental objects for dental applications.

The user is responsible for selecting the correct device settings.

### 2.1.3 Intended environment

The device is only permitted for use in closed rooms within the specified ambient conditions (see technical data in the Operating Instructions, "Documents of equal applicability", page 6) and in compliance with the requirements for safe installation (see requirements regarding the location site in the Operating Instructions, "Documents of equal applicability", page 6)

## 2.2 Operator's obligations

The operator is responsible for the safe operation of the device.

- ▶ Ensure compliance and control:
  - a. Intended use
  - b. Statutory or other safety and accident prevention regulations
- ▶ Only operate the device in a technically perfect condition, in a proper, safety-conscious and risk-conscious manner and while observing the Operating Instructions ("Documents of equal applicability", page 6).
- ▶ Keep these instructions and all documents of equal applicability complete, legible and accessible to the personnel at all times.

## 2.3 Personnel qualifications

- ▶ Ensure that the personnel assigned to work with the device have read and understood these instructions and all documents of equal applicability, in particular safety, maintenance and repair information, before starting work.
- ▶ Ensure that personnel is aware of hazards and safety equipment ("Working areas, potential hazards and safety measures on the device", page 11).
- ▶ Manage the responsibilities, competence and monitoring of staff.
- ▶ All work must be carried out by qualified technical staff only.
- ▶ Personnel to be trained should only be working with the device under the supervision of qualified technical staff.

### 2.4 Staff obligations

- ▶ Only operate the device in a technically perfect condition, in a proper, safety-conscious and risk-conscious manner and while observing the Operating Instructions ("Documents of equal applicability", page 6).
- ▶ Refrain from any process that could endanger staff or third parties.
- ▶ In the case of safety-related malfunctions, switch off the device immediately and allow the fault to be rectified by an authorized service partner.
- ▶ In all cases of doubt regarding the safety of the appliance, switch off the appliance and prevent further use.
- ▶ In addition to the overall documentation, legal or other safety and accident prevention regulations including applicable standards and guidelines of the country in which the device is operated must be complied with.
- ▶ Do not wear jewellery such as rings, bracelets or watches when working on the appliance, especially when cleaning the curing chamber to prevent cutting injuries and bruising.

### 2.5 Personal protective equipment

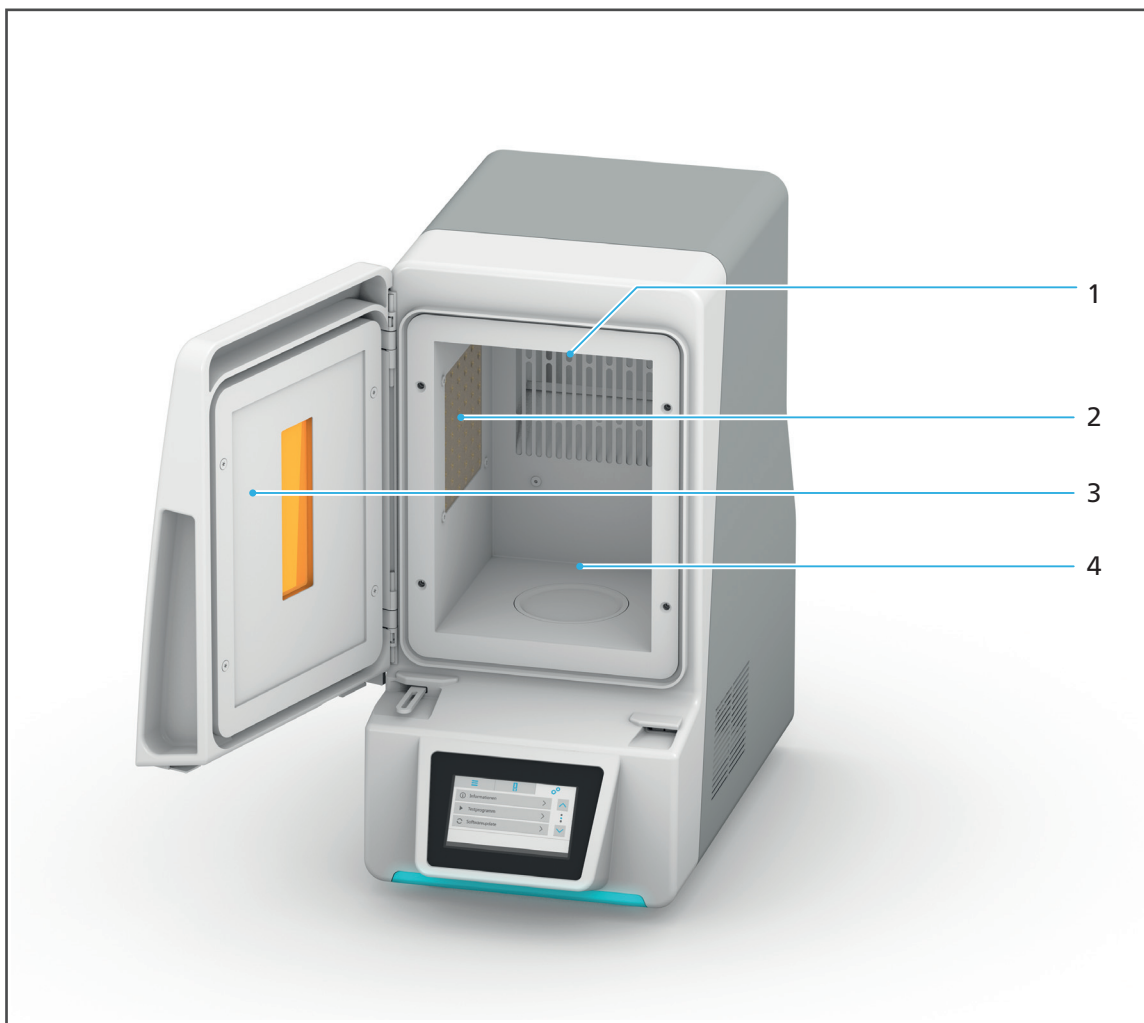
- ▶ If loud operating noise cannot be prevented, (sound level  $\geq 80$  dB(A)), use hearing protectors during operation.
- ▶ For the protection against material and cleaning agent vapours or other dental material particles while working with build platform and dental objects or during cleaning, wear protective masks (half mask with protection class FFP3), gloves and protective gear suitable for the respective material (see also Instructions for Use of the materials, "Documents of equal applicability", page 6).

## 2.6 Working areas, potential hazards and safety measures on the device

As an operator, you are working at the areas described below.

The device is built according to state-of-the-art technology and recognized technical safety regulations. Hazardous areas, which cannot be avoided by design, are equipped with appropriate protective devices. Nevertheless, if the machine is used improperly, there is a danger to life or risk of injury to the user or third parties. In addition the devices or printed dental objects may become damaged.

The working areas, their potential hazards and applicable safety measures are shown below:

No.	Working area	Tasks / Work	Type of hazard	Safety equipment	Protective effect
1	LEDs (top side)	Curing the materials	Light emissions	<ul style="list-style-type: none"> <li>Tinted viewing window in the door</li> </ul>	Protection against injury to the eyes
2	LED (left side)				
3	Door	Access to curing chamber	Mechanical hazards	<ul style="list-style-type: none"> <li>Safety switch: Access to the chamber during operation is not possible due to the locked door</li> <li>When the door is open, the program stops or the program cannot be started</li> </ul>	<ul style="list-style-type: none"> <li>Protection against burns</li> <li>Protection against crushing</li> </ul>
4	Curing chamber	<ul style="list-style-type: none"> <li>Inserting the build platform or material</li> <li>Cleaning when the device stands still</li> </ul>	Hot surfaces		

## 2.7 Safety instructions for potential hazard areas

### 2.7.1 Electromagnetic waves

Even though the valid standards are observed, it is possible that the device reacts to radio frequency interference or the operation of other devices in the immediate vicinity.

- ▶ Do not operate devices that generate strong electromagnetic waves in the same room as this device.
- ▶ Take appropriate measures in the case of radio interference, e.g. a new orientation, a new device position or electromagnetic shielding.
- ▶ Only use the original accessories recommended and supplied by the manufacturer. Failure to do so can result in increased interference emissions or decreased immunity of the device.

### 2.7.2 Light emissions

Looking straight into LEDs or reflective surfaces is unpleasant for the eye. Prolonged exposure can cause injury to the eyes.

In particular, this applies to people who work with this device or in its vicinity for long periods of time and to people who have undergone eye surgery.

- ▶ Make sure that people who are generally sensitive to light or take medication for light sensitivity or photosensitizing drugs are not exposed to the device light.
- ▶ Only view objects in the curing chamber through the tinted viewing window of the device.

### 2.7.3 Electrical safety

If covers are opened or parts are removed which are only accessible with tools, voltage-carrying components may be exposed. The plugs may also be under voltage. There is a risk of electric shock.

- ▶ Electrical work must only be carried out by your authorized service partner.
- ▶ Before connecting the device, check that the supply voltage and frequency are correct at the point of installation (see electrical data and the information on the type plate in the Operating Instructions, "Documents of equal applicability", page 6).
- ▶ To avoid injury to the user and for cooling reasons and fire protection, never remove the housing covers.
- ▶ For installation, observe the requirements of the manufacturer for house installation.
- ▶ To prevent the risk of electric shock, connect the device only to a power supply with a protective conductor. If the protective conductor is interrupted either inside or outside of the device or the protective conductor connection is loose, the device may be dangerous for the operator in the event of a fault or error. Intentional interruption is not permitted.
- ▶ The mains plug is used as a supply circuit disconnecting means. Always connect the mains plug to an easily accessible protective contact socket.
- ▶ Connect the grounded and freely accessible protective contact socket to a separately secured circuit.
- ▶ Make sure that the protective contact socket is equipped with a residual current circuit breaker (FI).

- ▶ Connect the device to a separately secured circuit or make sure that no devices are connected that cause severe supply voltage fluctuations when switched on. These fluctuations may interfere with the electronic controls and may cause failure of the system.
- ▶ Disconnect power before cleaning and maintenance work approved to be performed by the user.
- ▶ If you need to disconnect the unit from the power supply at a later time: Disconnect the plug from the protective-contact mains socket, not on the device.

### 2.7.4 Hazardous materials

Incorrect handling of the device, the accessories or non-compliance with the processes described in these Operating Instructions may lead to contact of the user with uncured material or the cleaning agent isopropanol (isopropyl alcohol). This can be detrimental to health.

Without corresponding safety precautions, there is a risk of skin irritation, allergic reactions and respiratory diseases.

If the cleaning agent isopropanol is used incorrectly, there is also a risk of explosion and fire.

- ▶ Observe the Instructions for Use of the materials used ("Documents of equal applicability", page 6).
- ▶ Wear protective gear ("Personal protective equipment", page 10).
- ▶ Make sure that the room is sufficiently ventilated when working with the materials.
- ▶ Only use mild, pH neutral cleaning agents for cleaning.

## 2.8 Safety information for the individual operating phases

### 2.8.1 Transport

- ▶ Transport the device only as described in the Operating Instructions (see requirements regarding transport and setting up in the Operating Instructions, "Documents of equal applicability", page 6).
- ▶ Transport and store the device only within the permissible temperatures and ambient conditions (see requirements for ambient conditions in the Operating Instructions, "Documents of equal applicability", page 6).

### 2.8.2 Initial operation

- ▶ Ensure that this device is only operated by authorized and trained specialists.
- ▶ If the device has been stored in a cold environment or at high humidity, maintain a drying or temperature adjustment time of approx. 1 hours (without voltage) at room temperature before initial operation.
- ▶ Before connecting the device, check that the supply voltage and frequency are correct at the point of installation (see electrical data in the Operating Instructions, "Documents of equal applicability", page 6).
- ▶ The requirements regarding the location site and ambient conditions must be observed (see requirements regarding the location site and ambient conditions in the Operating Instructions, "Documents of equal applicability", page 6).

### 2.8.3 Operation

- ▶ The device may only be operated by authorized and trained technical personnel.
- ▶ Keep unauthorized persons, such as patients, children and animals, away from the device.
- ▶ In all cases of doubt regarding the safety of the appliance, switch off the appliance and take suitable measures to prevent further use.
- ▶ Prior to connecting power or operation, check the device, the accessories and protective equipment for any damage.
- ▶ Do not use damaged, non-functioning equipment or accessories; instead notify your authorized service partner.
- ▶ In order to ensure the product safety and warranty services, the device must be exclusively operated with the original accessories from Ivoclar Vivadent, particularly the original power cord. The user bears the risk when using non-approved accessories.
- ▶ To ensure processing safety, Ivoclar Vivadent recommends using only approved materials.
- ▶ Never bypass the reliability equipment of the device or set it out of operation ("Working areas, potential hazards and safety measures on the device", page 11).
- ▶ Do not open the curing chamber whilst a program is in operation.
- ▶ Always place the build platform or object tray on the designated turntable.
- ▶ To avoid the risk of electric shock, ensure that no liquids or objects enter the ventilation slots.

- ▶ To prevent damage to the device and a reduction of the device performance, observe the cleaning requirements and cycles (see "Cleaning by the user", page 16 in these Short Instructions as well as the overview and instructions for the cleaning and maintenance work in the Operating Instructions, "Documents of equal applicability", page 6).
- ▶ Only operate the device unsupervised if the operating conditions for unsupervised operation described below are fulfilled.

### **Unsupervised operation:**

The device may be operated unsupervised, provided the national and local laws and provisions allow for such action and provided that they are observed. Furthermore, the requirements of the respective insurance company must be met.

- ▶ Never use the device if the curing chamber is heavily soiled.
- ▶ Protect the device against unauthorized access.

### **2.8.4 Cleaning by the user**

- ▶ Only clean as specified in the Operating Instructions and observe the related safety regulations (see overview and requirements regarding cleaning and maintenance work in the Operating Instructions, "Documents of equal applicability", page 6).

### **2.8.5 Improper maintenance, modifications and repairs**

Improper service and improper repairs or modifications will endanger patients and users and result in damage to the device and the final products.

Should you carry out any repairs or maintenance of the device or modifications or remove the housing without prior written consent by an authorized service partner, all warranty claims are void.

Unauthorized opening and removal of components can expose voltage-carrying components. The plugs may also be under voltage. There is a risk of electric shock!

- ▶ Maintenance as well as repairs of damaged safety equipment or machine parts must only be carried out by a service partner authorized by Ivoclar Vivadent.

### 3 Additional information

- Download link Operating Instructions PrograPrint System:  
<https://www.ivoclardigital.com/prograprint-manual>

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This device has been developed solely for use in dentistry. Setup and operation should be carried out strictly according to the Operating Instructions. Liability cannot be accepted for damages resulting from misuse or failure to observe the Instructions. The user is solely responsible for testing the apparatus for its suitability for any purpose not explicitly stated in the Instructions.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.