Tetric® N-Bond Self-Etch

Instructions for Use

Mode d'emploi

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Instructions for Use

Description

Tetric® N-Bond Self-Etch is a single-component, self-etching, light-curing, filled adhesive system mono frasco fotopolimerizable for enamel and dentin bonding.

Composition

Tetric N-Bond Self-Etch contains derivatives of bis-acrylamide, water, bis-methacrylamide dihydrogen phosphate, amino acid acrylamide, hydroxy alkyl methacrylamide, highly dispersed silicon dioxide, catalysts and stabilizers.

Indication

Adhesive for direct light-curing composite and compomer restorations.

Contraindication

- Do not use Tetric N-Bond Self-Etch if a patient is known to be allergic to any of the material's ingredients or if the stipulated working technique cannot be employed.
- Tetric N-Bond Self-Etch must not be used in combination with dual- or self-curing composites.
- Do not use Tetric N-Bond Self-Etch for direct pulp capping.

Side effects

In rare cases, ingredients of Tetric N-Bond Self-Etch may cause sensitization. In such cases, the product should cease to be used.

Interactions

Materials containing eugenol or clove oil may inhibit the polymerization of Tetric N-Bond Self-Etch.

Application

1. A dry operating field must be ensured. Ideally, a rubber dam should be used (e.g. the user-friendly OpraDam® Plus).

- In very deep cavities, areas close to the pulp should be selectively coated with a calcium hydroxide liner (e.g. ApexCal) and subsequently covered with pressure resistant cement (glass ionomer cement, e.g. Vivaglass Liner).
- Tetric N-Bond Self-Etch is available either in a bottle or the VivaPen.

VivaPen Delivery Form

- Unscrew the cap of the VivaPen.
- Attach the VivaPen brush cannula. By rotating the front end of the pen, the pre-bent cannula can be brought into the position desired for the application of the adhesive (see picture).

- By activating the click mechanism several times, the VivaPen brush cannula is wetted with Tetric N-Bond Self-Etch. Upon wetting the brush cannula, the yellow colour of Tetric N-Bond Self-Etch becomes visible.
- Subsequently, apply a thick layer of Tetric N-Bond Self-Etch on the enamel and dentin surfaces and brush in for at least 30 seconds. All cavity surfaces should be thoroughly coated. The total reaction time should be no shorter than 30 seconds.
 - Disperse excess amounts of Tetric N-Bond Self-Etch with a strong stream of air until there is no longer any movement of the material.
 - Polymerize Tetric N-Bond Self-Etch for 10 seconds at a light intensity of more than 500 mW/cm² (e.g. bluephase, bluephase C8, LEDition).
 - Apply the restorative material (e.g. Tetric N-Ceram) according to the instructions of the manufacturer.

Note on the VivaPen

- After each use, the VivaPen needs to be disinfected using disinfectant spray or wipes for hygienic reasons. Additionally, the VivaPen can be protected against contamination with the VivaPen sleeve.
- Discard the used VivaPen brush cannula and replace it with the original cap. A new VivaPen brush cannula must be attached for subsequent applications.
- Do not use oxidizing disinfectants.
- Do not immerse the VivaPen in disinfectant solutions.
- Do not use any other brush cannulas than VivaPen brush cannulas.

Warnings

- Tetric N-Bond Self-Etch is irritating. Avoid contact with skin, mucous membranes, and eyes. If the product comes in contact with the skin, rinse with water.
- If Tetric N-Bond Self-Etch comes in contact with the eyes, immediately rinse with copious amounts of water for at least 15 minutes and consult a physician/ophthalmologist.
- In rare cases, contact with skin may lead to a sensitization to methacrylates/methacrylamides.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates/methacrylamides.

Storage

- Storage temperature: 2-28 °C (36-82 °F).
- Close bottles/VivaPen immediately after usage. Exposure to light causes premature polymerization.
- Expiry date: See note on bottles, VivaPen and packaging

Keep material out of children's reach!

For use in dentistry only!

Date information prepared: 01/2010, Rev. 0

Manufacturer
Ivoclar Vivadent AG,
FL-9494 Schaan/Liechtenstein

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes.

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Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.