**Description**

Heliomolar® is a microfilled, radiopaque composite for the restorative therapy and cures with light in the wavelength range of 400–500 nm.

**Composition**

The monomer matrix is composed of Bis-GMA, urethane dimethacrylate and decandiol dimethacrylate (22 wt%). The fillers are composed of highly dispersed silicon dioxide, ytterbium tri-fluoride and copolymer (77.8 wt%). Additional contents are catalysts, stabilizers, and pigments (0.2 wt%). The total content of inorganic fillers is 46.0 vol% or 66.7 wt%. The particle size ranges from 0.04 to 0.2 µm.

**Indication**

- Anterior restorations (Classes III, IV)
- Class V restorations (cervical caries, root erosion, wedge-shaped defects)
- Restorations in the posterior region (Classes I and II)
- Reconstruction of traumatically damaged anterior teeth
- Veneering of discoloured anterior teeth
- Shape and shade corrections to improve the esthetics
- Splinting of mobile anterior teeth
- Extended fissure sealing
- Repair of facets
- Extraoral fabrication of inlays/onlays

**Contraindication**

The placement of Heliomolar restorations is contraindicated

- if a dry working field cannot be established or the prescribed application technique cannot be applied;
- if the patient is known to be allergic to any of the ingredients in Heliomolar.

**Side effects**

In individual cases, components of Heliomolar may lead to sensitization. The product must not be used in such cases. In order to exclude irritations of the pulp, areas close to the pulp must be protected with an appropriate pulp/dentin protection (selectively apply a calcium hydroxide-based preparation in areas close to the pulp and cover with a suitable cavity liner).

**Interactions**

Phenolic substances such as eugenol/clove oil inhibit the polymerization of materials based on methacrylate. Such materials should therefore not be used in conjunction with Heliomolar. Discolouration may occur in combination with cationic mouthwashes, plaque disclosing agents and chlorhexidine.

**Application****1. Shade selection**

Clean the teeth prior to shade determination. The shade is selected with the tooth still moist.

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## 2. Isolation

Adequate relative or absolute isolation using suitable auxiliaries, such as OptraGate® or OptraDam® Plus, is required.

## 3. Cavity preparation

The cavity is prepared according to the principles of the adhesive technique, i.e. by preserving as much of the tooth structure as possible. Do not prepare sharp internal angles or additional undercuts in caries-free areas. The geometry of the cavity is predetermined by the dimensions of the caries lesion or the old filling. Bevel enamel edges of anterior teeth. In the posterior region, only the sharp enamel edges should be lightly broken or rounded (finishing diamonds, 25–40 µm). Caries-free cervical defects are not prepared, only cleaned with pumice or other suitable cleaning pastes with the help of rubber cups or rotary brushes. Subsequently, rinse the cavity with water spray to remove all the residue and dry with water- and oil-free air.

## 4. Pulp protection / Base

Do not apply a base material when using an enamel/dentin bonding agent. Only cover very deep areas close to the pulp selectively with a calcium hydroxide material (e.g. ApexCal®) and subsequently use a pressure-resistant cement (e.g. a glass ionomer cement, such as Vivaglass® Liner). Do not cover other cavity walls, since they can be used to support the bond with an enamel/dentin adhesive.

## 5. Placement of matrix / interdental wedge

Use a wrap around matrix for cavities affecting the proximal area or a sectional matrix and wedge it.

## 6. Conditioning / Application of the bonding agent

Condition and apply the bonding agent according to the Instructions for Use of the product in use. Ivoclar Vivadent recommends using the bonding agent Syntac® or ExciTE® F (both in conjunction with phosphoric acid etching), or Adhese® Universal (in the self-etching mode or in conjunction with phosphoric acid etching).

## 7. Application of Heliomolar

- Apply Heliomolar in layers of max. 2 mm thickness and adapt it with a suitable instrument (e.g. OptraSculpt).
- Sufficient exposure to the curing light prevents incomplete polymerization. For the recommendations regarding exposure time and light intensity see table 1.
- When using a metal matrix, additionally polymerize the composite material from the vestibular or lingual/palatal aspect after removing the matrix, if no Bluephase polymerization light is used or the light guide cannot be ideally positioned, e.g. distant to composite or diverging scattering angle.

## 8. Finishing / Checking the occlusion / Polishing

After polymerization, remove excess material with suitable finishers or fine diamonds. Remove proximal excess with diamond carbide finishers, finishing strips or flexible finishing discs. Check the occlusion and articulation and grind in the necessary adjustments to prevent premature contacts or undesired articulation paths on the surface of the restoration. Use silicone polishers (e.g. OptraPol®) as well as polishing discs and polishing strips to polish the restoration to a high gloss.

## Additional notes

1. In the case of repairs, additional Heliomolar can be directly applied to the polymerized material. If the Heliomolar restoration has already been polished, it has to be roughened and wetted with Heliobond before a new layer of Heliomolar can be applied.
2. Heliomolar can also be used to repair prosthetic restorations made of SR Nexco® and SR Chromasit®. If ceramic restorations are repaired, a bonding agent (Monobond Plus) needs to be applied.
3. Use Heliomolar at room temperature.  
Cold material may be difficult to dispense.
4.  For single use only. If Heliomolar is directly applied from the Cavifil in the mouth of the patient, the Cavifil should only be used for one patient due to hygienic reasons (prevention of cross-contamination between patients).
5. Do not use oxidizing disinfectants to disinfect syringes and Cavifils.

## Warning

Contact of the uncured Heliomolar material with the skin, mucous membrane and eyes must be avoided. Unpolymerized Heliomolar may have a slight irritating effect and may lead to a sensitization against methacrylates. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

## Shelf life and storage

- Storage temperature 2–28 °C (36–82 °F)
- Close syringes/Cavifils immediately after usage. Exposure to light causes premature polymerization.
- Do not use Heliomolar after the indicated date of expiration.
- Expiry date: see note on Cavifils, syringes and packages.

## Keep out of the reach of children!

## For use in dentistry only.

The product has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for its suitability and use for any purpose not explicitly stated in the Instructions.

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**Ivoclar Vivadent Marketing Ltd.**
**Ivoclar Vivadent AB**
**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**