IPS Empress® Direct

The highly esthetic composite

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Dear users

Thank you for choosing our product.

IPS Empress Direct is designed for direct restorative treatment procedures. It is particularly suitable for restoring anterior teeth. The material has been developed on the basis of many years of experience and expertise. The coordinated shade system offers ideal solutions for many different clinical colour situations and individual optical characteristics.

IPS Empress Direct materials have a long processing time and therefore allow you to work without pressure under optimum light conditions. A wide selection of Dentin materials helps you to obtain the desired level of opacity in the dentin core. They are also suitable for creating mamelons. More translucent Enamel materials are available for imitating enamel areas. In addition, highly translucent materials are provided for accentuating certain characteristics in the final layer.

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Trans 20

20%*

Customization of the incisal third in middle-aged patients

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Trans 30

30%*

Customization of the incisal third in young patients

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Trans Opal

45%*

Creation of an opalescent effect in the incisal third

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Three special flowable Effect materials complement the sculptable products. Together with IPS Empress Direct Color, these materials enable creative practitioners to give every restoration a highly individualized appearance.

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Brown
Braun

White

- Imitation of spotted enamel
- Simulation of the "halo" effect
- Accentuation of cusps and ridges

Honey / Ochre

- Enhancement of the dentin core
- Accentuation of fissures
- Imitation of individual enamel characteristics

Blue / Violet / Grey

- Simulation of translucent enamel areas
- Imitation of opalescent areas

Brown

- Imitation of stained pits, fissures and enamel cracks

The following layering techniques and clinical case presentations show you how easy it is to create your very own dental masterpieces with IPS Empress Direct. We hope you will enjoy working with IPS Empress Direct.

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Simple layering technique

Anterior tooth built up with two shades

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EN

- 1 The palatal enamel wall is rebuilt with IPS Empress Direct Enamel A1.
- 2 The proximal walls are rebuilt with Enamel A1.
- 3 The dentin core is shaped and the mamelons on the incisal edge are created.
- 4 A final layer of Enamel A1 is placed.

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Layering technique with Effect materials

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2

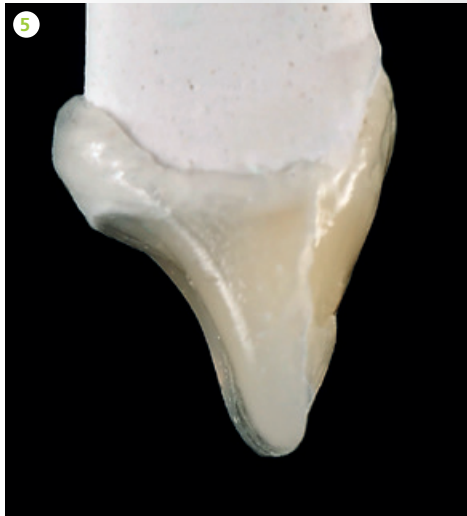
EN

- 1 The palatal enamel wall is rebuilt with IPS Empress Direct Trans 30.
- 2 The proximal walls are rebuilt with Trans 30.
- 3 The dentin core is shaped and the mamelons on the incisal edge are created with Dentin A2.
- 4 IPS Empress Direct Effect Trans Opal is applied on the incisal edge, between and on the mamelons.
- 5 Side view: The different materials from the palatal to the labial aspect (enamel-dentin) together with the incorporated Trans Opal are easy to distinguish.
- 6 A final layer of IPS Empress Direct Trans 30 is applied.

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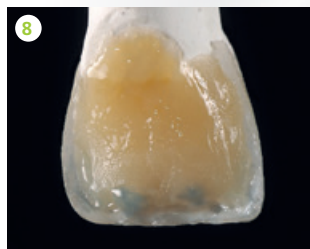
3

EN

- 1 The palatal enamel wall is rebuilt with IPS Empress Direct Trans 20.
- 2 The proximal walls are rebuilt with Trans 30
- 3 The dentin core is shaped up to the margin of the incisal edge with Dentin A3
- 4 IPS Empress Direct Color Ochre is placed on the cervical areas.
- 5 The colour is spread in the cervical areas.
- 6 IPS Empress Direct Color White is applied on the incisal edge.
- 7 The colour is carefully worked into the incisal edge.
- 8 IPS Empress Direct Color Blue is placed between the lobes in the incisal edge.
- 9 A final layer of IPS Empress Direct Trans 20 is placed.

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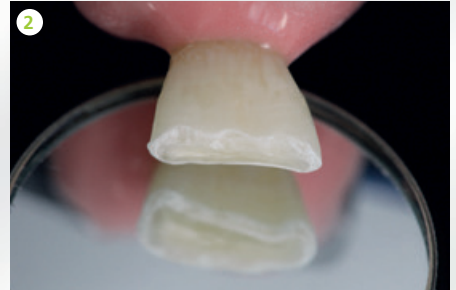
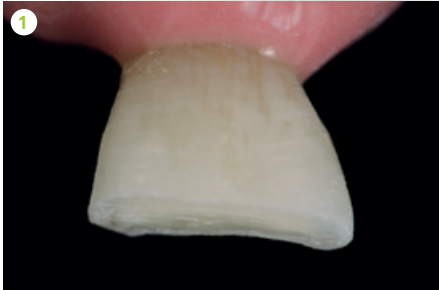
Young patient, Fractured incisal edge

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

EN

- 1 Preoperative situation: labial view of the fractured incisal edge after preparation
- 2 Preparation with scalloped margin
- 3 The adhesive is applied.
- 4 A silicone matrix is placed and the first layer of IPS Empress Direct Effect Trans 30 is applied.
- 5 Two-thirds of the enamel wall is rebuilt with IPS Empress Enamel A2 using OptraSculpt Ball. IPS Empress Direct Trans 30 is applied up to the incisal edge.
- 6 The silicone matrix is removed and the proximal walls are rebuilt with IPS Empress Direct Enamel A2 using OptraSculpt Chisel.
- 7 The dentin core is rebuilt and the mamelons are shaped with IPS Empress Direct Dentin A3 using OptraSculpt Point.

- 8 The incisal edge is coated with IPS Empress Direct Effect Trans Opal.
- 9 The labial tooth wall with the exception of the incisal edge is rebuilt with IPS Empress Direct Enamel A2 and the restoration is smoothed with OptraSculpt Pad.
- 10 The incisal edge is coated with a final layer of IPS Empress Direct Trans 30 and then smoothed.
- 11 The restoration is finished and polished with OptraPol.
- 12 Completed restoration. The mamelon and "halo" effects and the translucent zone are clearly visible.

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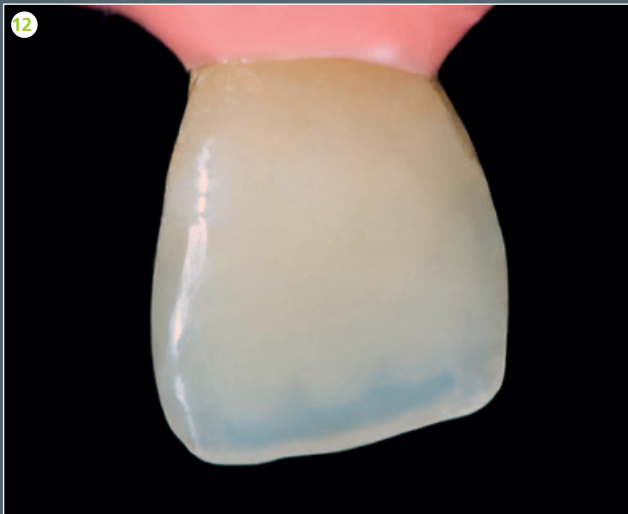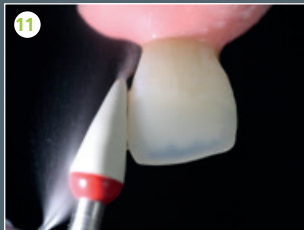Mature patient, Incisal edge built up with enamel crack

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

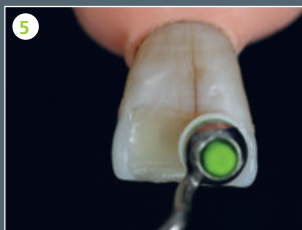
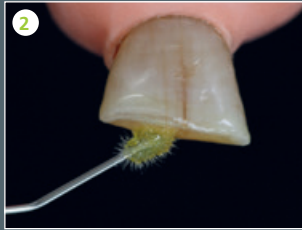
Dr Anton Lebedenko, Ivoclar Vivadent, Liechtenstein

EN

- 1 Preoperative situation: fractured incisal edge. View of the prepared tooth from the labial aspect (note the enamel crack down the middle)
- 2 The adhesive is applied.
- 3 A silicone matrix is placed to help rebuild the palatal side with IPS Empress Direct Effect Trans 30 and IPS Empress Direct Enamel Bleach XL.
- 4 The proximal walls are rebuilt with IPS Empress Direct Enamel Bleach XL using OptraSculpt Chisel.
- 5 The dentin core is rebuilt with IPS Empress Direct Dentin (V45) and shaped using OptraSculpt Pad. Mamelons do not have to be created in mature patients.
- 6 A final layer of IPS Empress Direct Enamel Bleach XL is placed.
- 7 Fine crack lines are formed with OptraSculpt Chisel in order to imitate the existing crack.
- 8 IPS Empress Direct Color Brown and Ochre are placed in the created grooves.
- 9 The grooves are closed up again using OptraSculpt Pad. As a result, the colours are embedded below the final enamel layer. The tooth surface is smoothed.
- 10 The restoration is finished and polished using OptraPol.
- 11 Completed restoration. The imitated crack optimally blends in with the overall appearance of the tooth.

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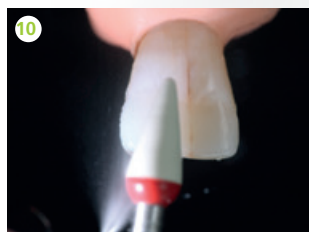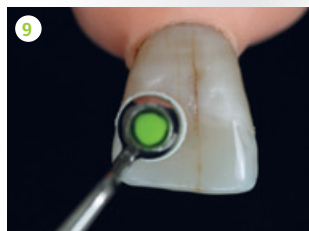Ivoclar Vivadent, Liechtenstein

EN

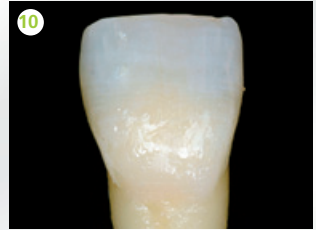
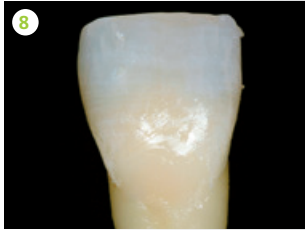
- 1 Preoperative situation:
Class V cavity; facial view
- 2 Disto-facial view
- 3 Preparation line marked
- 4 Completed preparation
- 5 Tooth structure after conditioning and
application and polymerization of bonding
agent
- 6 Dentin material is applied
- 7 Disto-facial view
- 8 First increment of the appropriate Enamel
shade is applied
- 9 Disto-facial view
- 10 Second increment of Enamel material is
applied, restoring the tooth to its original
shape.
- 11 Disto-facial view
- 12 Mesio-facial view
- 13 Restoration completed and polished.

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
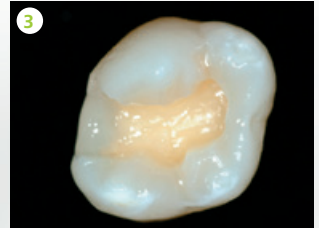
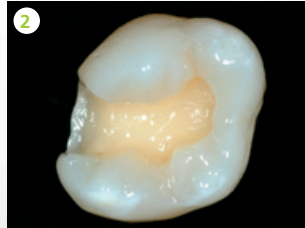
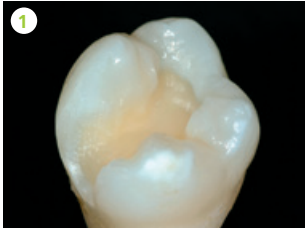
7

EN

- 1 Preoperative situation: Class II
- 2 Tooth structure after conditioning and application and polymerization of bonding agent
- 3 The first increment of composite (Enamel material) is placed in the area of the proximal box.
- 4 Mesial view. The first increment is placed horizontally.
- 5 The second increment of composite (Enamel material) is added.
- 6 The second increment is placed obliquely, extending from the occlusal preparation margin to slightly short of the opposite wall.
- 7 The third increment of Enamel material completes the proximal wall.
- 8 Mesial view


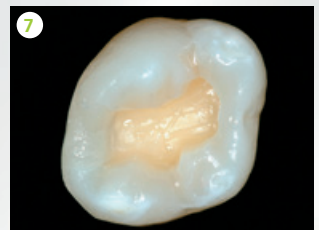
- 9 The dentin core is rebuilt with Dentin material; third layer.
- 10 Completed mesio-palatal cusp rebuilt with Enamel material
- 11 Disto-buccal view
- 12 Completed disto-buccal cusp rebuilt with Enamel material
- 13 Mesial view
- 14 Restoration of the last remaining cusp
- 15 Completed restoration after pit & fissure characterization using IPS Empress Direct Color Brown and after polishing.

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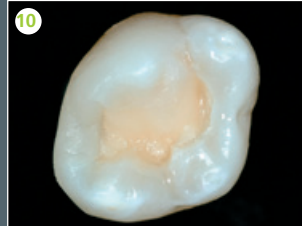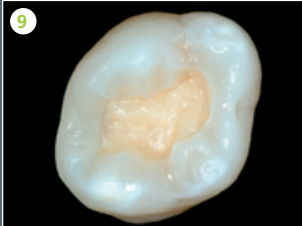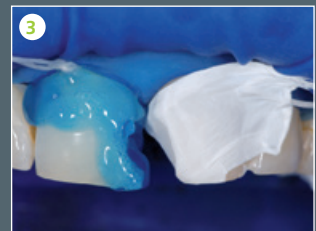
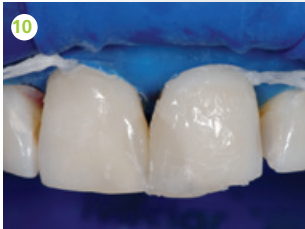
Clinical case
Klinischer Fall

EN

- 1 Preoperative situation: Fracture of the mesial edge of tooth 21 as well as esthetically unsatisfactory Class IV fillings in teeth 11 and 21 Class IV filling in tooth 11 (mesially).
- 2 Situation after excavation and creation of a wide bevel along the cavity margins
- 3 Tooth 11 is etched with phosphoric acid. Tooth 21 is protected with Teflon tape during the procedure.
- 4 Application of adhesive in tooth 11
- 5 Placement of the palatal enamel layer in tooth 11 with the help of a silicone matrix
- 6 Completed restorations in tooth 11. Tooth 21 has already been conditioned.
- 7 Reconstruction of the palatal enamel wall using a silicone matrix and matrix band
- 8 Polymerized enamel shell in the palatal area of tooth 21
- 9 Reconstruction of the dentin core in tooth 21
- 10 Reconstruction of the facial wall with Enamel and Trans 30 material
- 11 Restorations after finishing and polishing
- 12 Final result after rehydration of the teeth

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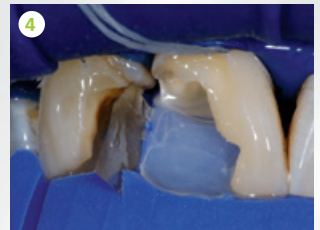
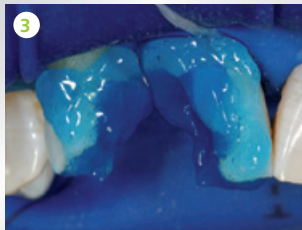
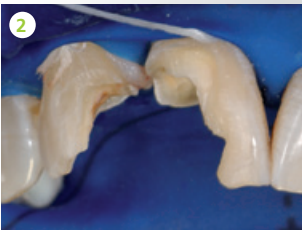
2





EN

- 1 Preoperative situation: large, defective restorations in teeth 12 (mesially) and 11 (distally)
- 2 Extensive lesions after excavation and beveling of the margins
- 3 Tooth structure conditioned with phosphoric acid
- 4 The palatal enamel wall is rebuilt against a silicone matrix using IPS Empress Direct Enamel.
- 5 Polymerized palatal enamel shell after removal of the silicone matrix
- 6 Reconstruction of the anatomically shaped dentin core, customized by placing thin strips of IPS Empress Direct Color White
- 7 Reconstruction of the distal surface with a lighter Enamel shade
- 8 Situation after the reconstruction of a large part of the facial surface; some space is left for the Trans 30 material
- 9 The restoration was completed with a layer of Trans 30 and subsequently polished. Tooth 12 was restored according to the same principle. View immediately after the restoration had been completed.
- 10 The restoration was completed with a layer of Trans 30 and subsequently polished. Final result after 24 hours and rehydration of the dental hard tissue.

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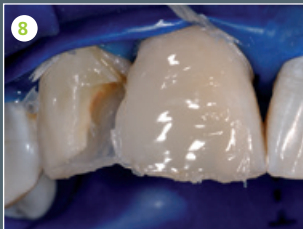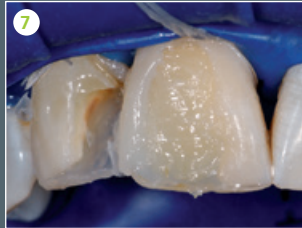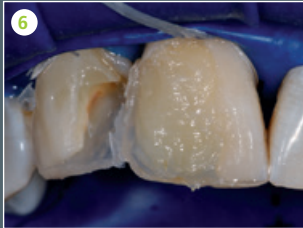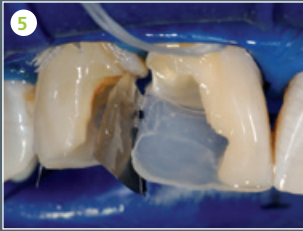Restoration with opalescent effect

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Dr Markus Lenhard, Switzerland

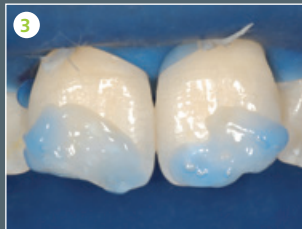
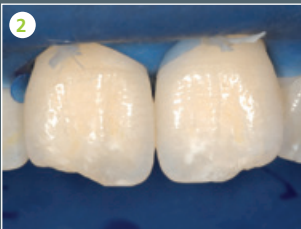
3

Clinical case
Klinischer Fall

EN


- 1 Preoperative situation: Fracture of the distal incisal edges of the central incisors
- 2 Situation after a convex bevel has been prepared
- 3 Enamel-etch technique
- 4 Application of the bonding agent
- 5 Completed build-up of the palatal wall
- 6 Application of Dentin material
- 7 Application of opalescent material between and along the sides of the mammelons
- 8 Placement of the facial Enamel layer
- 9 View directly after completion of the restoration
- 10 Postoperative situation after 2 weeks

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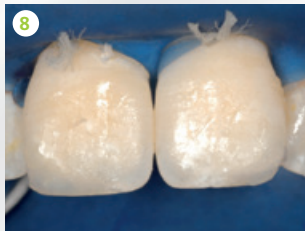4

EN

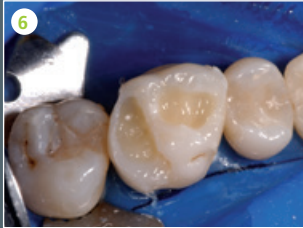
- 1 Preoperative situation: Recurrent caries under the composite restorations in tooth 16 (om and odp)
- 2 Situation after removal of the restorations. The disto-palatal cusp had to be removed.
- 3 Sectional matrix in place
- 4 Reconstruction of the proximal enamel walls using IPS Empress Direct Enamel
- 5 Situation after polymerization of the Enamel material. The Class II cavities were reshaped into two Class I cavities.
- 6 Dentin material was used to rebuild the dentin core.
- 7 Completed restoration: The occlusal aspects were built up with Enamel material and subsequently polished.

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What the experts say

Experten-Statements

“Anyone can master the art of producing highly esthetic composite resin restorations in just a few steps with IPS Empress® Direct.”

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Dr Markus Lenhard,

Switzerland/Schweiz

“IPS Empress® Direct is easy to polish. Restorations have a beautiful high-gloss finish. The Trans Opal material allows you to successfully imitate the opalescent qualities of natural teeth.”

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Lieferformen

Refills

Sculptable consistency available in 1 x 3-g syringe or 10 x 0.2-g Cavifils

IPS Empress Direct Dentin	A1 A2 A3 A3.5 A4 B1 B2 C3 D2 IVA5 IVA6 Bleach L Dentin Bleach XL Dentin
IPS Empress Direct Enamel	A1 A2 A3 A3.5 A4 B1 B2 B3 B4 C1 C2 C3 D2 D3 Bleach L Enamel Bleach XL Enamel
IPS Empress Direct Translucent	Trans 20 Trans 30 Trans Opal

Flowable consistency available in 1 x 1.8-g syringe (including 5 tips)

IPS Empress Direct Effect	Trans 30 Trans Opal Bleach XL
IPS Empress Direct Opaque	Opaque

IPS Empress Direct Color available in 1 x 1-g syringe (including 5 tips)

IPS Empress Direct Color	white honey blue violet ochre brown grey
---------------------------------	--

IPS Empress Direct Color Assortment

637898AN

7 x 1 g IPS Empress Direct Color
1 x 1.8 g IPS Empress Direct Opaque
Various accessories

Accessories

Tips 20 x 0.4 mm 634847

IPS Empress Direct Color

Tips 20 x 0.9 mm 634849

IPS Empress Direct Effect
IPS Empress Direct Opaque

IPS Empress Direct Pad 628369

IPS Empress Direct System Kits

SYRINGES

686071AN

2 x 3 g Dentin (A2, A3)
 3 x 3 g Enamel (A1, A2, A3)
 1 x 3 g Trans 30
 1 x 1.8 g Effect Trans Opal
 1 x 1.8 g Opaque
 1 x Autoclavable Shade Guide Dentin
 1 x Autoclavable Shade Guide Enamel/Translucent
 Various accessories

686072AN

2 x 3 g Dentin (A2, A3)
 3 x 3 g Enamel (A1, A2, A3)
 1 x 3 g Trans 30
 1 x 1.8 g Effect Trans Opal
 1 x 1.8 g Opaque
 1 x OpraSculpt Pad Instrument
 1 x OpraSculpt Pad Refill (4 mm)
 1 x Autoclavable Shade Guide Dentin
 1 x Autoclavable Shade Guide Enamel/Translucent
 Various accessories

CAVIFILS

686093AN

20 x 0.2 g Dentin (A2, A3)
 30 x 0.2 g Enamel (A1, A2, A3)
 10 x 0.2 g Trans 30
 1 x 1.8 g Effect Trans Opal
 1 x 1.8 g Opaque
 1 x Autoclavable Shade Guide Dentin
 1 x Autoclavable Shade Guide Enamel/Translucent
 Various accessories

686095AN

20 x 0.2 g Dentin (A2, A3)
 30 x 0.2 g Enamel (A1, A2, A3)
 10 x 0.2 g Trans 30
 1 x 1.8 g Effect Trans Opal
 1 x 1.8 g Opaque
 1 x OpraSculpt Pad Instrument
 1 x OpraSculpt Pad Refill (4 mm)
 1 x Autoclavable Shade Guide Dentin
 1 x Autoclavable Shade Guide Enamel/Translucent
 Various accessories

Autoclavable Shade Guide
 Enamel/Translucent
 627215

Autoclavable Shade Guide
 Dentin
 627214

Ivoclar Vivadent AG
Bendererstr. 2
9494 Schaan
Liechtenstein
Tel. +423 235 35 35
Fax +423 235 33 60
www.ivoclarvivadent.com

628376/EN-DE/2019-09-20

ivoclar
vivadent[®]
passion vision innovation